# **WORK INSTRUCTIONS**

## 3.3.3 Screening Blood for Acidic and Neutral Drug Compounds

#### REFERENCE MATERIAL

#### Mixed Standard Solution

- Add 20µL of each compound.
- ✓ Total volume is 10mL.
- Solution is stable for 6-months @ room temperature.
- Record Preparation on standard log.

1mg/mL Stock Solution
Acetaminophen
Butalbital
Carbamazepine
Carisoprodol
Meprobamate
Phenobarbital
Secobarbital

### Internal Standard Mix

#### **Proadifen**

olice Her of copy 1201 A

on the Her of copy 120 Prepare 10mg Dilute 'N Shoot Standard with 10mL of DI water.

Prepared standard is stable for 4 months when stored at 4°C.

### Working Internal Standard Solution [20ng/μL]

Add 200µL each Proadifer and 1mg/mL Aprobarbital stock solutions to 10mL volumetric ball flask. QS with DI water.

Solution is stable for three months when stored at room temperature.

# **WORK INSTRUCTIONS**

## 3.3.3 Screening Blood for Acidic and Neutral Drug Compounds

