ACROSEP

RESIN SEPARATOR

ACROSEP is manufactured from selected chemicals of guaranteed purity including sodium alginate and is effectively used as a separating medium of plaster / resin and flask / plaster surfaces.

ADVANTAGES

- It produces a durable surface film which does not peel off during trial closures.
- Application of ACROSEP ensures easy separation of surfaces so that the final cured acrylic has an extremely smooth and shiny surface.
- The product has an exceptionally long shelf life.
- It can also be used to coat the inside of flasks before pouring, which enables them to be emptied easily after use and avoids their serious distortion by prolonged hammerings.

DIRECTIONS FOR USE

- Ensure that all the wax is completely boiled out. Any wax residue will affect the result. Flushing the surface of the plaster with a neutral detergent before application is useful precaution.
- Before the plaster has cooled completely paint the surface with one or two coats of ACROSEP using either a brush or cotton pellet.
- Allow the film to dry leaving a glossy surface and pack the flask in the usual manner.

SPECIAL NOTES

- When using the ACROSEP to prevent plaster sticking to the flask pour the plaster immediately after application.
- Use the ACROSEP via its polyethylene nozzle. Do not return any surplus to the bottle as this will cause contamination.
- Precipitation may occur in the bottle, but this will not affect the effectiveness.
- Personal protective equipment (PPE) such as gloves, face masks and safety eyewear should always be worn.

PACKAGE

Bottle of 300 gram

STORAGE

Store in a cool place away from direct sunlight (4-25°C.) (Shelf life: 3 years from date of manufacture)

Some products referenced in the present IFU may be classified as hazardous according to GHS. Always familiarize yourself with the safety data sheets available at:
http://www.gceurope.com
They can also be obtained from your supplier.

Last revised: 04/2016

<p>MANUFACTURED by</p> <p>GC ASAHl CORP.</p> <p>1-3-3 Kachigawa-cho, Kasugai, Aichi 486-0945, Japan</p> <p>DISTRIBUTED by</p> <p>GC CORPORATION</p> <p>76-1 Hasunuma-cho, Itabashi-ku, Tokyo 174-8585, Japan</p> <p>EU : GC EUROPE N.V.</p> <p>Researchpark Haasrode-Leuven 1240, Interleuvenlaan 33, B-3001 Leuven, Belgium TEL: +32 16 74 10 00</p>	<p>GC ASIA DENTAL PTE. LTD.</p> <p>11 Tampines Concourse, #03-05, Singapore 528729</p> <p>TEL: +65 6546 7588</p> <p>GC AUSTRALASIA DENTAL PTY. LTD.</p> <p>1753 Botany Rd, Banksmeadow, NSW 2019, Australia</p> <p>TEL: +61 2 9316 4499</p>
<p>PRINTED IN JAPAN</p>	

The GC logo is a stylized representation of the letters G and C. The G is formed by a thick black line that curves around the top and left sides of the letter, ending in a small tail. The C is formed by a similar thick black line that curves around the top and right sides, also ending in a small tail. The two letters are positioned close together, with the G slightly overlapping the C.

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