

•No Reporting of suspect case to Maine CDC •No **Required To Submit** Laboratory Specimen • Information on requisition must include: suspected organism, patient name, DOB, date of collection, specimen source or type, submitter name and contact information. **Required Information** •Important: all specimens must be labeled with patient name and be accompanied by a HETL requisition. • Preferred: Nasopharyngeal swab, Throat swab, Nasal wash Specimen •Other Acceptable specimens: Lower respiratory specimens (bronchial wash), Nasal aspirate, Sputum Requirements •ONLY synthetic-tipped swabs (Polyester, Rayon, Dacron) with plastic shafts should be used. • Specimens should be submitted in viral transport media provided in the kit. •Store specimens at 2-8°C until ready for shipping. Specimens should be submitted to lab as soon as **Collection Instructions** possible. Use wet ice or cold packs to maintain 2-8°C during delivery. •PCR results are generally available within 24 hours after arrival at HETL. Turn Around Time • Specimens received without a form and clinical details or with insufficient material to analyze. Unacceptable • Specimens in poorly labeled or leaking containers will not be tested. Conditions • Do not use wooden shaft or calcium alginate swabs • Do not freeze specimens for PCR testing • Parainfluenza virus RNA detected or Parainfluenza virus RNA not detected • Parainfluenza virus positive samples will be subtyped Results • All results will be reported only to submitter as stated on requisition via mail or fax. •\$110.00 Laboratory Testing Fee •87899 Applicable CPT Codes Bacteriology at 207-287-1704, Virology, Respiratory, Arbovirus, and Serology at 207-287-1722 Molecular Biology, Sequencing, and Foodborne at 207-287-5769 Additional Information • Mycobacterium and Rabies at 207-287-1706 •CT/GC at 207-287-6244

Detection of Parainfluenza virus by Real Time PCR

Laboratory Submission Information Sheet