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ⒼⒷ Instructions for Use

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D-76307 Karlsbad-Ittersbach

Contents

General information.....	28
Preliminary remarks	28
Declaration of conformity	28
Purpose	28
Indication	28
Contraindication	29
Signs and symbols	29
Liability	29
Safety notes	30
Product and delivery overview	31
Inspection of delivery	31
Features of the Libra wheelchair	31
Labelling and type plates	32
Preparing the operating condition	32
Using your wheelchair	35
Getting in and out.....	35
Getting in and out from the side.....	35
Getting in and out from the front.....	36
Dismount the side parts	36
Swing-up the foot plates	36
Moving ahead with the wheelchair	37
Pushing and slowing down with the help of an assistant.....	37
Independent moving and applying the brakes.....	38
Locking brake.....	39
Transport and folding.....	40
Adjustments	41
Locking brake	41
Lower leg length	41

Adjustment backrest angle.....	42
Adjustment seat depth.....	43
Advice on product care	45
Cleaning	45
Transfer/further use.....	45
Checking the wheelchair by user	45
Service notes for specialist retailers	46
Disposal.....	46
Technical data	47
Service record.....	48
Warranty	49

General information

Preliminary remarks

Thank you for having chosen a wheelchair from Tomtar.

The design, functionality and quality of this modern wheelchair won't disappoint you.

Before operating the unit for the very first time, please read this manual carefully.

You will obtain vital safety instructions and valuable advice on the right use and maintenance of the wheelchair. If you have questions or require additional information, please contact the specialist retailer that supplied the wheelchair.

These instructions for use contain all information required to adjust and operate the Libra wheelchair. The most remarkable feature about the Libra wheelchair is its low weight. The frame is made of high-strength aluminium.

Declaration of conformity

The DIETZ GmbH declares under sole responsibility that the Libra wheelchair described here conforms to the EU directive 93/42 for medical products.

Purpose

The Libra wheelchair can be used both indoors and outdoors. The product is only to be used for self-transportation and for transporting people.

The wheelchair is designed for a maximum load of 120 kg. You can verify this information on the label.

Indication

Inability to walk or severe walking disability as a result of

- paralysis,
- loss of limbs,
- meromelia/limb deformation,
- joint contractures/damage to joints (not on both arms),
- other ailments.

Contraindication

Use of the wheelchair is not suited for

- perception disorders, severe disturbances of equilibrium,
- loss of limbs on both arms,
- joint contractures/damage to joints on both arms,
- inability to sit,
- reduced or insufficient eyesight.

Signs and symbols

	Caution! Indicates information that is particularly relevant to safety. Follow the instructions for use.
	Important! Indicates information that is particularly useful in the respective context.

Liability

DIETZ GmbH Reha-Produkte only assumes liability when products are used in accordance with the specified conditions and for the aforementioned purposes. We recommend using the products properly and maintaining them in compliance with these instructions. DIETZ GmbH Reha-Produkte does not assume liability for damage caused by components and spare parts not approved by DIETZ GmbH Reha-Produkte. Repairs may only be carried out by authorised specialist retailers or by the manufacturer themselves.

Safety notes

In order to avoid toppling and dangerous situations, you should practice safe handling of your new wheelchair on a flat and familiar surface. In such case, an assistant is recommended.

- Never leave children and youngsters unattended in the wheelchair.
- Prior to using the wheelchair, please check whether all parts attached have been fastened properly.
- The first attempts at moving the wheelchair should take place on a level and manageable surface. Familiarise yourself intensively with the braking and driving behaviour of the wheelchair when moving straight ahead and turning corners.
- Please note that padded parts can become hot in direct sunlight and lead to skin injury on contact. Therefore, cover these parts or protect the wheelchair from direct sunlight.
- The wheelchair may not be misused for transporting several persons or loads. Observe the specific function of the wheelchair.
- Never use your wheelchair while under the influence of alcohol or other substances that may influence your ability to react or your physical and mental receptiveness.
- Please note that redistribution of weight due to body movements or loading the wheelchair can increase the risk of tipping over.
- By all means, use ramps to avoid obstacles.
- Cross grooves, tracks and similar impassibilities in a right angle (90°), if you cannot move around them.
- There is a danger of tipping over to the rear starting at an angle of 10°.
- Avoid wheeling against obstacles (steps, kerbs) at full speed or moving down steps.
- Never stand on the foot plates to get in and out.
- Take note of a sufficient profile depth of your tyres (>1mm).
- Observe the road traffic regulations when on the road.
- If possible, wear bright, noticeable clothes. You will then be seen better by other traffic participants.
- Do not use high-pressure cleaning devices or harsh, corrosive chemicals to clean the wheelchair!

Product and delivery overview

Inspection of delivery

The wheelchair is completely packed in a cardboard box. Please keep the packaging, if possible, it may serve for storing the wheelchair at a later time.

Upon receipt, immediately check the content for completeness and sound condition. The content consists of:

- 1 packaging
- 1 wheelchair assembled
- 1 headrest
- 1 footrests (pair)
- 1 manual

Features of the Libra wheelchair

Labelling and type plates

Type and manufacturer plates are mounted to the cross-brace under the seat.

Plates must remain readable. Immediately replace unreadable or missing plates. The type plate is very important for identifying the product. It may not be removed.

Preparing the operating condition

Generally, your wheelchair is delivered completely assembled, folded and with the drive wheels mounted. To make it ready for use, some easy and simple hand movements are required.

Position yourself beside the wheelchair. Press down the cross-brace with your flat hand.

Danger of getting trapped!

Watch your fingers when folding and setting up the wheelchair and only grasp the parts illustrated.

Make sure that the cross-brace fits proper to the socket.

Adjust the locking strap in horizontal position and tighten the locking lever firmly.


Plug the headrest into the mounting rails of the backrest.

Push the headrest down to the limit.

Plug the legrests into the guide bushing.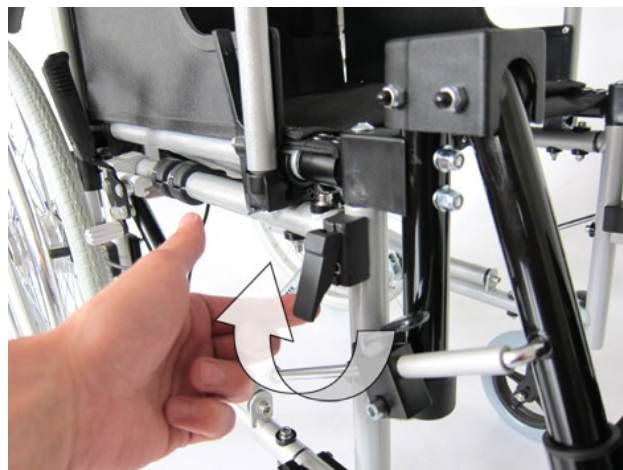

For releasing just pull the release lever and pull the legrest upwards. The legrest is fully unlocked if the release lever is positioned horizontal.

The footrests are easily to swing-up.
For folding the wheelchair the footrests must be positioned vertically.

Using your wheelchair

Getting in and out

Getting in and out from the side

The wheelchair allows you to easily get in and out. The foot plates and leg supports can be turned sideways, the side parts can be removed upwards.

Place the wheelchair and the current or future seating surface as close beside one another as possible.

Should you wish to transfer from another wheelchair or another seating to the wheelchair, make sure that the wheels are secured by applying the locking brake.

Secure the wheelchair against accidental rolling away by adjusting the brake lever on the right and left-hand side.

Lift up the brake lever from the arrest position and turn the lever down to the rear. In that way, you provide a flat surface for the transfer.

Pivot the side part to the rear.

Fold the foot plates upwards.


Pivot the leg supports towards the inside or exterior (if necessary, of both wheelchairs when you wish to transfer from one wheelchair to the other).

Now slide sideways onto the seating surface.

Make sure you are sitting in the seat as far back as possible.

Finally, pivot the side part and the leg supports back into their home position.

Pull the levers of the locking brake into the operating position.

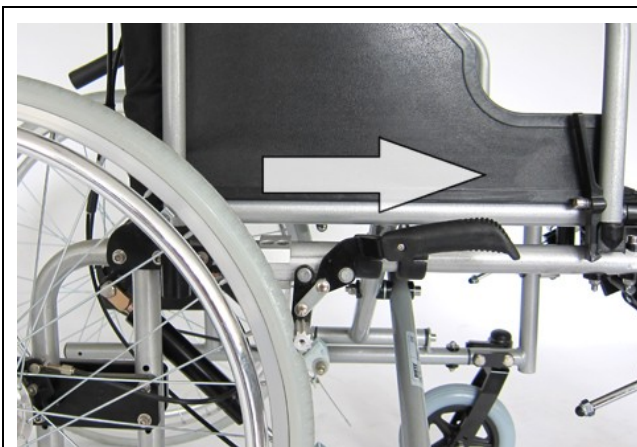Locking brake adjustment:

Push the brake lever into driving position. The brake lever now is positioned horizontally and allows easy transfer.

Getting in and out from the front

Fold the foot plates upwards and pivot the leg supports towards the exterior. (Description see above.)

Place the wheelchair and the current or future seating surface as close to the front of one another as possible.

Should you wish to transfer from another wheelchair or another seating to the wheelchair, make sure that the wheels are secured by applying the locking brake.

Now slide onto the seating surface by twisting your body. Make sure you are sitting in the seat as far back as possible.

Subsequently, swivel the leg supports back in centre position, fold down the foot plates and rest your feet thereon.

Dismount the side parts

Push the release lever and pull the side rest upwards.

By assembling the side rests the release lever snaps automatically.

Swing-up the footrests

The footrests are easily to swing-up. For folding the wheelchair the footrests must be positioned vertically.

Moving ahead with the wheelchair

Pushing and slowing down with the help of an assistant

The assistant moves ahead the wheelchair by pushing the designated pushing handles.

Important!

The assistant must be physically and mentally capable of slowing down the wheelchair with own strength. The locking brake can be used as parking brake.

Attendant brake (optional)

An attendant brake is available as an option.

By activating the brake lever the wheelchair can be stopped smooth and easily.

The attendant brake can be fixed by using the locking mechanism. Please note the attendant brake does not replace the locking brake.

Independent moving and applying the brakes

Practice safe handling of your new wheelchair on a flat surface with the assistance of an aide, until you get accustomed to it. You may only participate in public traffic with sufficient travelling experience. Be especially careful when moving on inclined surfaces and when overcoming obstacles.

The wheelchair is set in motion and slowed down via the push rim.

To move the wheelchair ahead, put your hand on the push rim, whereas only your thumb and bent index finger touch the push rim.

The other fingers are closed to a fist and do not touch the push rim.

To slow down, let the push rim glide between your thumb and bent index finger and slow it down with a tightening grip until the wheelchair comes to a stop.

Caution!

When moving the wheelchair, make sure that you do not touch the top of the tyre with your thumb.


Danger of getting jammed between tyre and side part/arm rest!

Locking brake


Apply the locking brake mechanism when the wheelchair stands still (tyre brake). For this purpose, push the lever forward until it locks by itself.

Now the wheelchair keeps locked, and you can transfer to another seating surface or just park.

You can release the locking brake by retracting the lever.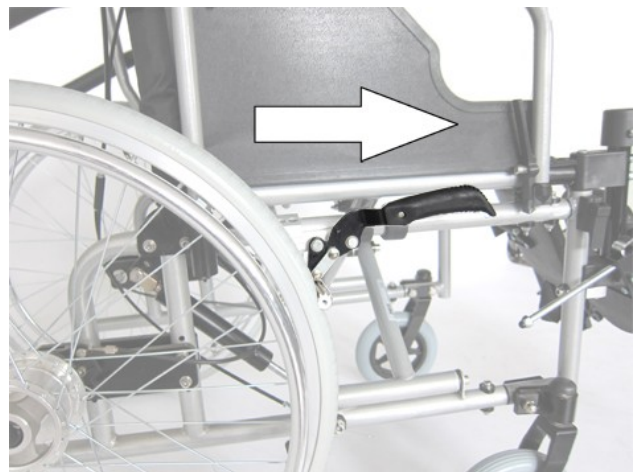

In a braking position

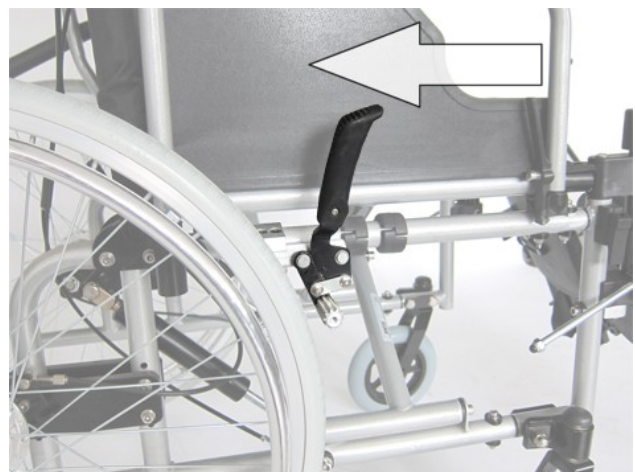In a non-braking position

To steer the wheelchair into a left and right direction, slow down the push rim on that side you would like to direct it.

To turn the wheelchair on the spot, one hand pushes forward, while at the same time the other hand pushes backwards.

Transport and folding

For transportation in a passenger car or simply to clear space, your wheelchair can be folded up easily. It then only takes little space and can be easily stowed away.

Caution!

The wheelchair is not approved as a seat in vehicles such as disabled person transportation vehicle.

Leave the wheelchair.

Fold the foot plates upwards.
Remove the leg supports, if applicable.

Pull the covering of the seating surface upwards in centre.

The wheelchair will then fold up automatically.

To set up the wheelchair, please observe the instructions given under the chapter "Preparing the operating condition".

Adjustments

Only trained specialists should adjust brakes. However, should you have sufficient experience; you may also carry out this adjustment yourself.

Caution!

Incorrectly adjusted brakes may result in life-endangering consequences.

Locking brake

The distance between the brake block and the surface of the tyre must be 12mm at the narrowest point when the brake mechanism is opened to its maximum.

To adjust the distance, loosen the two screws.

Move the brake into the correct position.

Afterwards tighten the screws again and check the brake function.

If the brake is correct adjusted, the wheelchair should be immovable.

Lower leg length

The adjustment is right, when your thighs are horizontal with both feet resting on the footplates.

Loosen the screw and move the foot plate into a position that is comfortable for you. Please note the bores inside the tube preset the different steps.

Tighten the screw firmly afterwards.

Adjustment backrest angle

The backrest can be adjusted variable supported by two gas springs. Angle range = 0° - 70°

To avoid that the wheelchair rolls away please lock both locking brakes at a first step. Hold both grip handles firmly and pull the adjustment lever at the right side. Then push the backrest downwards until the ideal position is reached and release the lever at the same time. Now the backrest is adjusted tight. In contrast pull the release lever and push the backrest in driving position. The two gas springs support by tilting up.

Adjustment seat depth

The seat depth can be adjusted successively. Please note the adjustment should be done by healthcare professionals only. As a first step you have to remove the side rests. (shown p. 12).

Please unscrew the screws of the seat cushion (10pcs.) Most important is that the front most two screws (left / right) should not be unscrewed.

As a second step please loosen the Velcro between the seat cushion and the seat support.

The seat support now can be pulled out horizontal step by step. Please note the bores inside the tube preset the different steps.

<p>If the optimal position is reached please fasten the screws of the seat cushion in the front most position first. Afterwards please fasten all the remaining screws equally.</p>	
<p>At the next step the front part from the side frame will be adjusted in order to match the already fixed seat cushion. Please fold the wheelchair first and unscrew all the six screws of the front part of the side frame.</p>	
<p>After unscrewing and disassembling all the screws you can pull out each side in driving position. Please note the different bores at the inner horizontal tube preset the adjustment.</p>	
<p>The optimal positioning of the side frames is reached if the bushings used for the foot rests end with the seat cushion. As a last step please fasten all the screws from the side frame.</p>	

Advice on product care

Cleaning

- The frame parts of the wheelchair can be wiped using a damp cloth.
- In case of heavy soiling, use a mild cleaning agent. To clean the wheels use a damp brush with plastic bristles (do not use a steel brush!).
- The padding materials can be washed up to 40°C with mild soap solution and sanitised with disinfecting agents on an alcohol basis, where required.

Do not use a high-pressure or vapour cleaning device!

Transfer / further use

The wheelchair is suitable for further use. When the wheelchair is transferred to / reused by another person, ensure that all technical documentation necessary for its safe use is given to the new user.

The wheelchair must be cleaned, disinfected, examined by a specialist retailer and approved.

Checking the wheelchair by user

Prior to each use, please check

- that the tyres are in order and have sufficient air pressure (given optional pneumatic tyres). Regularly check the profile depth of the tyres once a month. The profile depth should measure at least 1mm. Defective or worn tyres must be replaced.
- the operational reliability of the brakes. If the brake is in place, the wheelchair should be immovable. If the brake is released, the wheelchair should be easy to push without any grinding sounds. Should you notice any unfamiliar braking behaviour, immediately inform your specialist retailer and do not use the wheelchair any longer.
- the sound condition and cleanliness of the seat and backrest padding. If these are soiled, you can clean them in accordance with the instructions for cleaning. If they are damaged, contact your specialist retailer.
- all mobile parts for smooth movement and low-noise functioning. Squeaking noises indicate damage and should be examined with regard to their causes by a specialist retailer.
- all screw couplings and bolted connections are tight.

Service notes for specialist retailers

We recommend having your whole wheelchair inspected by a specialist retailer once a year. In case of faults and defects on your wheelchair, return it to your specialist retailer immediately for repair.

On request, Tomtar will provide specialist retailers with required information and documents for repair and service.

In case of queries, please contact the Tomtar Customer Service at phone No. +49 (0) 7248/ 9186 – 0.

Use the enclosed service record to have an overview of the service status of your wheelchair.

Disposal

If your wheelchair is not used any more and is to be disposed of, please enquire from local disposal companies about the applicable regulations.

Technical data

Libra		
Measurements and weights		
Seat width cm	42	45
Seat height cm	50	
Seat depth cm	40 - 46	
Seat angle °	2	
Overall width cm	61	64
Overall length cm	122	
Overall height cm	107	
Backrest height cm	91 (including headrest)	
Pushing handle height cm	97,5 – 109,5	
Armrest height cm	71,5	
Armrest width cm	5	
Armrest length cm	36	
Empty weight kg	24	
Max. load capacity kg	120	
Tyres		
Drive wheels	24x1 3/8	
Steering wheels	7x1 3/4	

Service record

(Please use, if possible, the service codes provided on the bottom of this page for filling in the columns)

When												
What												
Reflectors, lateral												
Reflectors, rear												
Drum brake												
Occupant brake lever												
Assistant brake lever												
Pushing handles												
Side part, complete												
Seat cover												
Backrest cover												
Front/steering wheels												
Rear/drive wheels												
Tyre pressure (given optional pneumatic tyres)												
Tyre cover												
Push rim												
Frame, general												
Who												

C = checked S = substituted R = repaired

Warranty

This warranty covers all defects in the product that are shown to be caused by material or production defects. The warranty period is 24 months from receipt of the notification of readiness for despatch or from delivery at the latest. Our warranty obligations do not include damage caused by natural wear and tear, with intent, by negligent or improper operation or use. The same applies to the use of inappropriate care products, lubricants or grease. In the case of complaints, please return the product together with the warranty slip completely filled in (see below).

Warranty slip

Product:

Specialist retailer

Send to:
TOMTAR
Descostrasse 10
D-76307 Karlsbad-Ittersbach