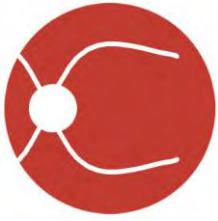
IDx-DR Client User Manual

Software Version 2
2018-11-30

| | |
|---|-----|
| IDx-DR Client User Manual (en)..... | 1 |
| Uživatelská příručka IDx-DR Client (cs)..... | 36 |
| IDx-DR Client-Benutzerhandbuch (de)..... | 71 |
| Manual del usuario de IDx-DR Client (es)..... | 107 |
| IDx-DR Client -ohjelmiston käyttöopas (fi)..... | 141 |
| Manuel de l'utilisateur IDx-DR Client (fr) | 176 |
| Manuale dell'utente di IDx-DR Client (it)..... | 210 |
| Gebruikershandleiding voor IDx-DR Client (nl)..... | 245 |
| Instrukcja obsługi systemu IDx-DR Client (pl) | 282 |
| Användarhandbok till IDx-DR Client (sv)..... | 319 |
| IDx-DR Client Kullanıcı Kılavuzu (tr)..... | 354 |

IDx Technologies Inc.
2300 Oakdale Blvd Coralville, IA 52241

IDx-DR Client User Manual

Software Version 2
2018-11-30

ENG-DRCLv2EU-S00140, Revision A
en

Note: The information in this manual only applies to the IDx-DR Client, the marked version. It does not apply to earlier versions. Due to continuing product innovation, specifications in this manual are subject to change without notice.

© 2018 IDx Technologies Inc. All rights reserved.

Information contained in this document is proprietary to IDx Technologies, Inc. No part of this publication may be reproduced, stored in a retrieval system, or transmitted in any form, in whole or in part, by any means electronic, mechanical or otherwise, including photocopying and recording, for any purpose without written permission.

Table of Contents

| | | |
|----------|---|-----------|
| 1 | <i>Manual Information</i> | 4 |
| 1.1 | Revision History..... | 4 |
| 1.2 | Purpose | 4 |
| 1.3 | Intended Audience..... | 4 |
| 1.4 | How to obtain a paper copy of this manual | 4 |
| 1.5 | Glossary of Symbols | 4 |
| 2 | <i>Warnings and Precautions</i> | 5 |
| 2.1 | Prerequisites | 5 |
| 2.2 | Warnings | 5 |
| 3 | <i>Product Information</i> | 6 |
| 3.1 | Product Contents..... | 6 |
| 3.2 | Principles of Operation..... | 6 |
| 4 | <i>Installation</i> | 7 |
| 4.1 | System Requirements | 7 |
| 4.2 | Getting Started..... | 7 |
| 4.3 | Installation..... | 7 |
| 4.4 | Registration..... | 8 |
| 4.5 | Exam Selection Tool Configuration..... | 8 |
| 4.6 | Disk Output Tool Configuration..... | 13 |
| 4.7 | Configure Image Source..... | 16 |
| 4.8 | Updates | 16 |
| 4.9 | Accessing the manual and viewing the product label..... | 18 |
| 5 | <i>Standard Use</i> | 19 |
| 5.1 | Submitting Retinal Images for Interpretation..... | 19 |
| 5.2 | Command Line Submission..... | 28 |
| 5.3 | Service and Maintenance | 30 |
| 6 | <i>Advanced Options</i> | 31 |
| 6.1 | Advanced configuration | 31 |
| 6.2 | Viewing IDx-DR Client logging information | 33 |
| 7 | <i>IDx-DR Troubleshooting</i> | 34 |
| 8 | <i>Reference Information</i> | 35 |
| 9 | <i>Software Licensing Information</i> | 36 |

1 Manual Information

1.1 Revision History

| Rev. | Date | Revision Description |
|------|------------|----------------------|
| A | 2018-11-30 | Manual Creation |

1.2 Purpose

This manual contains the instructions necessary to operate the IDx-DR Client software device in accordance with its function and intended use.

1.3 Intended Audience

This manual is intended for any person who uses, maintains, or troubleshoots this device.

1.4 How to obtain a paper copy of this manual

Contact IDx at support@eyediagnosis.net to request a paper copy of the manual. Please provide your organization name, full address, the product for which you are requesting a manual, and the version of the software. IDx will mail you a printed version of the manual via the postal service.

1.5 Glossary of Symbols

Warning - Indicates a potentially hazardous situation associated with the use or misuse of the device which, if not avoided, could result in serious injury, death, or serious other adverse reaction.

Consult Manual Before Use

2 Warnings and Precautions

2.1 Prerequisites

In order to write software to interact with IDx-DR Client safely and according to the intended purpose the following prerequisites must be met:

- You should read and understand the IDx-DR Warnings and Indications Manual in addition to this manual.
- You must read and understand the Intended Use, Warnings, and Operating Instructions.
- You should have a general understanding of the use of a person computer, running one of the operating systems in the Reference Information section below.
- IDx-DR Client should be installed according to the instructions in Section 4 of this User Manual.

2.2 Warnings

The following warnings apply to all aspects of software interacting with IDx-DR Client:

- Users should regularly ensure that IDx-DR Client software is operating on a computer that is free of viruses or malware and is updated with the latest security patches.

3 Product Information

3.1 Product Contents

IDx-DR Client.

3.2 Principles of Operation

After installation, the IDx-DR Client software connects to the IDx-DR Service and Analysis system. To submit exams to the device, retinal images are selected in the IDx-DR Client. In order to process the exam, IDx-DR Client needs to be connected to the internet. The IDx-DR Client sends the exam securely over the internet to IDx servers for processing. A result report and associated recommendation is then immediately generated, after processing, at the point of care by the IDx-DR Client. The results with the clinical decision can then be viewed in the IDx-DR Client. To perform an analysis, the IDx-DR Client must be authorized with a registration code which must be obtained from IDx and the user must have an active IDx-DR account.

4 Installation

Warning: IDx recommends that installation and system changes be performed by individuals familiar with the IT systems on which the IDx-DR Client is running.

4.1 System Requirements

To use the IDx-DR device you will need the following:

1. A computer running a Microsoft Windows Operating System
2. A working internet connection

Specific computer hardware requirements can be found in Section 8 of this User Manual.

4.2 Getting Started

Download the IDx-DR Client installation package based upon information provided by your IDx representative.

Please see associated camera manufacturer instructions for camera configurations and settings.

Inspect the camera lens and clean if applicable in accordance with the appropriate camera manual.

Configure the exam site so that minimal light is present. Best practices for inducing darkness at an exam site include:

- Turn off lights
- Close blinds or black out windows

4.3 Installation

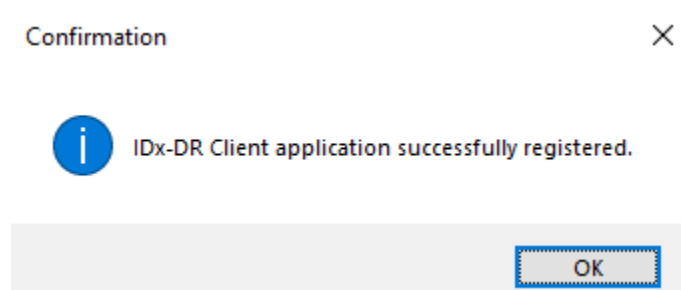
1. After downloading the installation package, use the Windows explorer to browse to the location on the hard-disk where the IDx-DR installation package has been saved.
2. Double-click the installation package.
3. Select the destination directory where the IDx-DR Client should be installed.
 - a. The destination directory must be writable and have sufficient space available.
4. Follow the installer prompts to install the software.

4.4 Registration

A registration code from IDx is required. Without registering the Client, it cannot be used to obtain results and will not function. To obtain a code, contact IDx at info@eyediagnosis.net or contact your IDx representative.

After installation, the IDx-DR Client can be started by double-clicking the “**IDxDRClient.exe**” file in the installation folder. The client will start up and ask for a registration code.

Fill in the registration code and press “OK” once finished. An internet connection is required for the IDx-DR Client to verify the registration code. Upon successful registration the following prompt is shown:

Press “Ok” to start the application. The first time the client is started, a copy of the electronic manual is also shown on screen.

4.5 Exam Selection Tool Configuration

The IDx-DR Client provides the Exam Selection Tool, which allows grouping images from the same patient into exams. There are two ways to group images into an exam:

- Using DICOM images
- Using the filename to group images

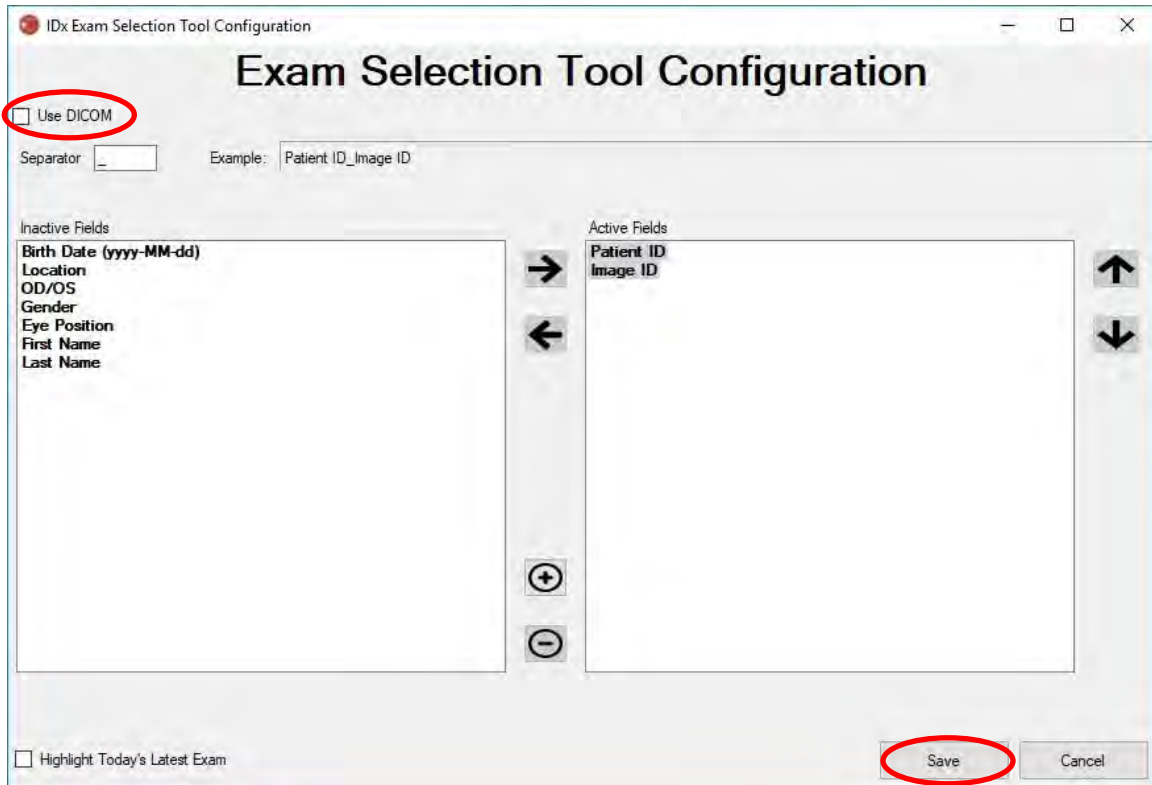
In both cases, the Patient ID is used to group images from the same patient. For the second option, a unique Image ID needs to be available.

When the IDx-DR Client is opened for the first time, the method of defining exams needs to be configured. Configuration directions for both methods of grouping images are below.

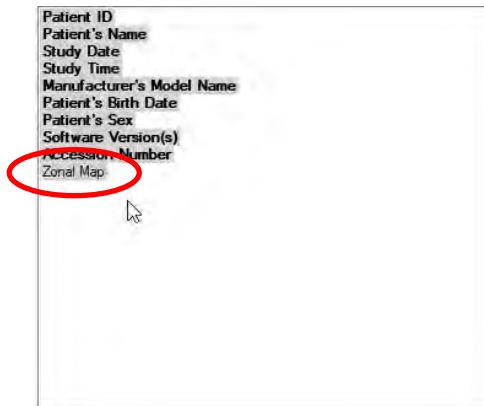
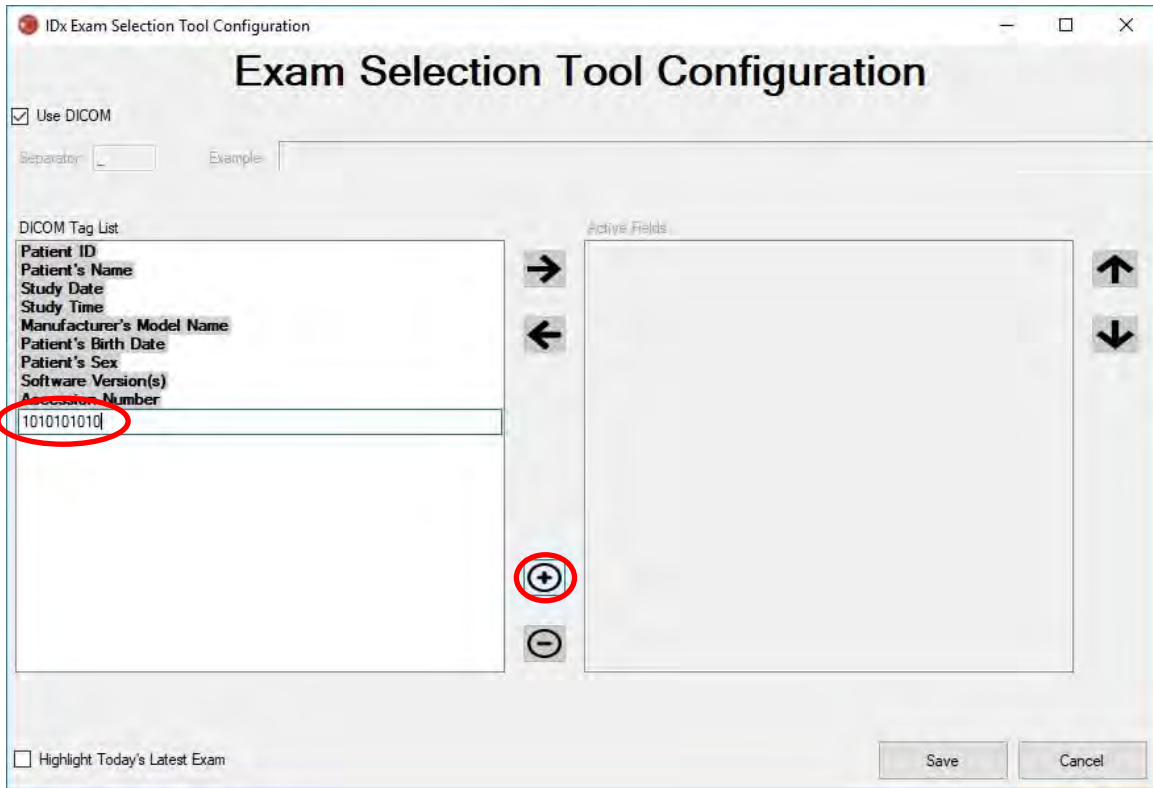
1. DICOM Based Exam Selection: To use DICOM images, simply check the “Use DICOM” checkbox in the upper left corner of the Exam Selection Tool Configuration window shown below and press the “Save” button. The Exam Selection Tool will

now look for DICOM images in the input directory and use the information in the DICOM files to group them.

NOTE: IDx-DR will only accept DICOM exams containing JPEG images.

2. If your DICOM images contains a field ID that is not in the “DICOM Tag List” list, use the “+” button to add a field ID. If the field ID is recognized it will automatically show as the name of the field.

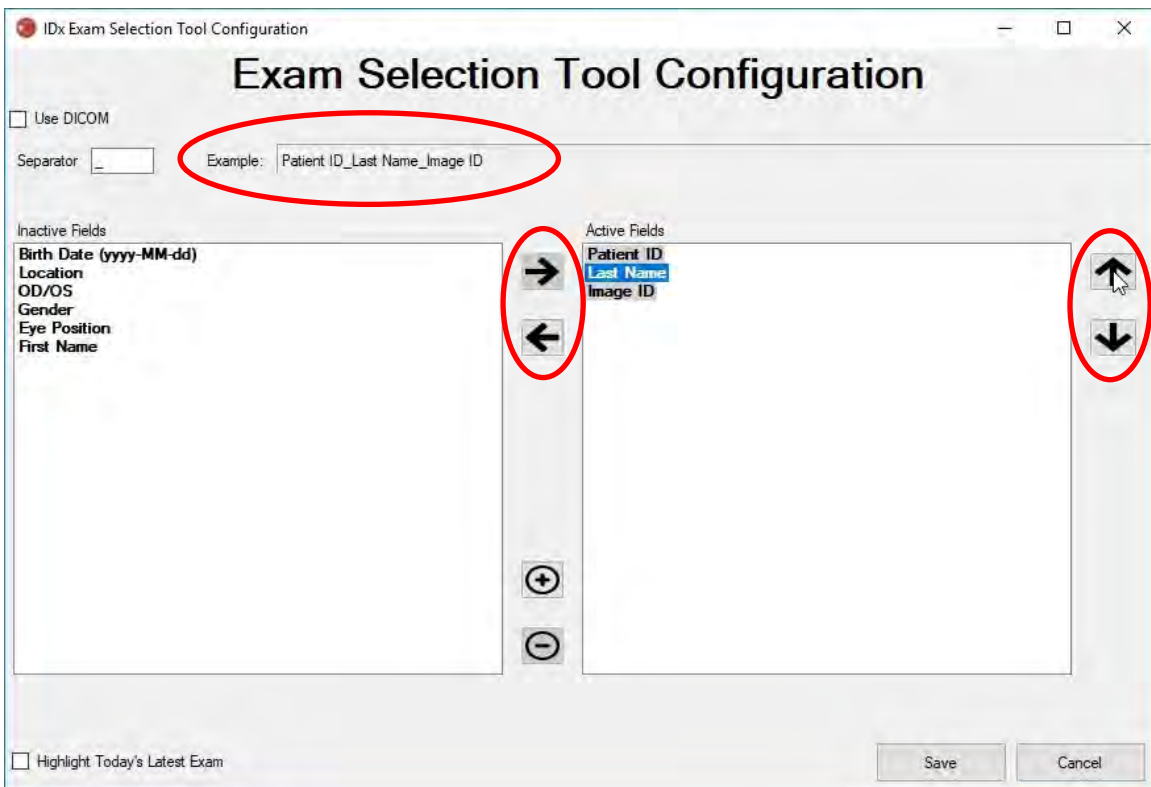
To continue with DICOM based configuration, skip to Item 5. To configure the filename based grouping, continue with Item 3.

3. Filename Based Exam Selection: Filename based grouping assumes that the filenames of the images presented to the software follow a predefined structure where the various information elements in the filename are separated using a consistent separation character. For example:

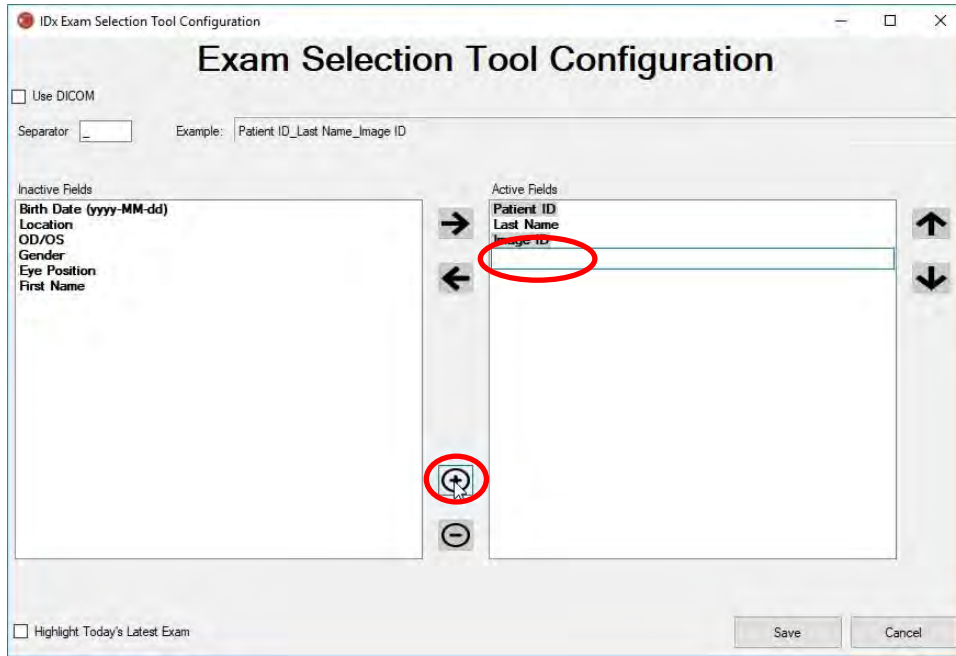
firstname_lastname_dob_patientid_imageid.jpg. In this example the separator is “_” (the underscore symbol).

Move all the fields that are present in the image filename to the Active Fields box in the Exam Selection Tool Configuration window. To achieve this, select an individual field and use the **left/right arrow buttons** to move the field between the Active and Inactive Fields windows (see graphic below). Make sure no unintended fields remain in the Active Fields window. The minimum elements required are Patient ID and Image ID. Without that information, filename based grouping will not work.

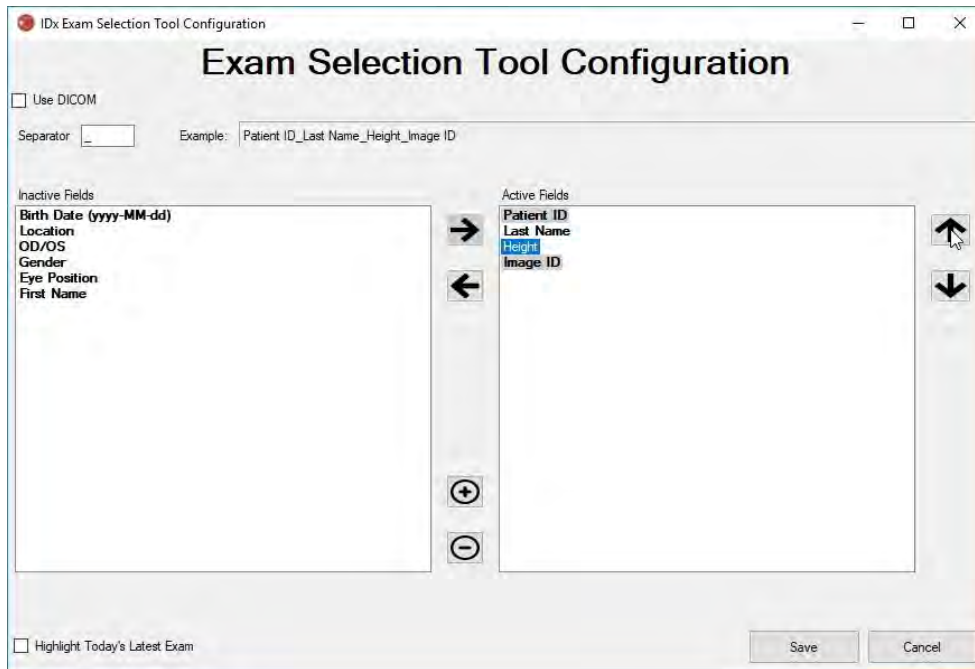
Next, select individual fields in the Active Fields window and use the **up/down arrow buttons** (on the right side of the window) to order the fields appropriately based on image filename convention. The filename convention, as it corresponds to the “Active Fields” box, is generated in the “Example” section of the Exam Selection Tool Configuration window (shown below).

4. If your filename contains unique fields that are not in the “Inactive Fields” list, use the “+” button to add a unique field. For example, if “height” was included in the filename:
 firstname_lastname_dob_patientid_height_imageid.jpg

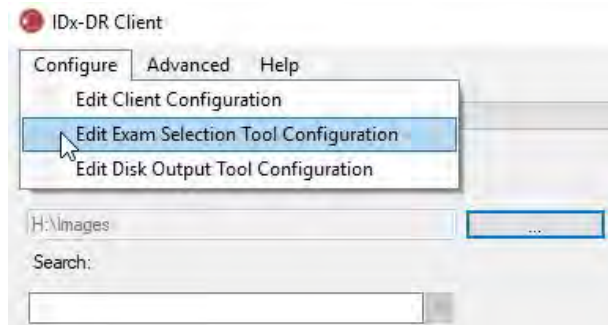
Enter the name of the field and move it to the correct position:

5. Selecting the check box “Highlight Today’s Latest Exam” will bold the text of exams from the current day and grey out the exams not from that day in the Exam Selection Tool.

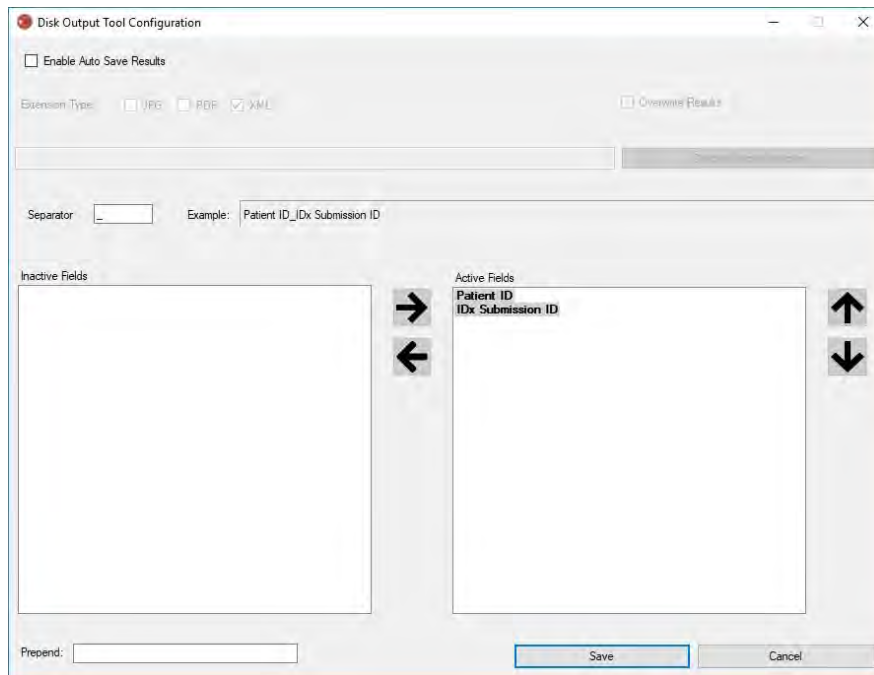
Press the “Save” button to finish configuring the Exam Selection.

6. If the Exam Selection Tool Configuration ever needs to be changed, select the “Edit Exam Selection Tool Configuration” option under the “Configure” dropdown menu (shown below) and revisit the steps in this section.

4.6 Disk Output Tool Configuration

The IDx-DR Client provides the Disk Output Tool, which allows users to save exam analysis reports.

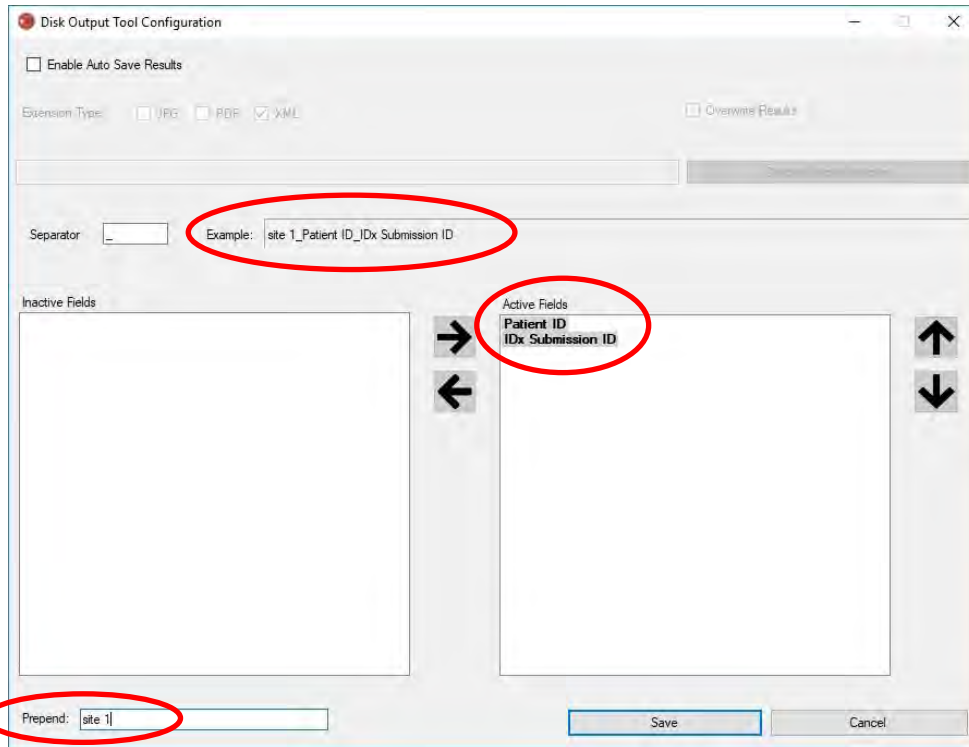
In the previously shown configuration window, the following configuration options are available (from top to bottom):

- “Enable Auto Save Results” – When this box is checked, all analysis reports will automatically save.
 - “Extension Type” – This allows the selection of the report format (PDF, JPEG, and/or XML).”
 - “Overwrite Results” – When this box is checked, only the most recent exam analysis report per visit will be saved.
 - “Choose Output Directory” – This allows the selection of the directory where exam analysis reports will be automatically saved.
- “Separator” – This allows for the definition of a character used as a separator for the various elements in the filename.
- “Prepend” – This allows for the definition of a fixed text to be added to the beginning of the filename.

To begin configuring the Disk Output Tool, select “Enable Auto Save Results” if you want the exam analysis reports to be automatically saved. If exam analysis reports are automatically saved you must select the extension type, check “Overwrite Results” if you want the results to be overwritten and choose the desired output directory.

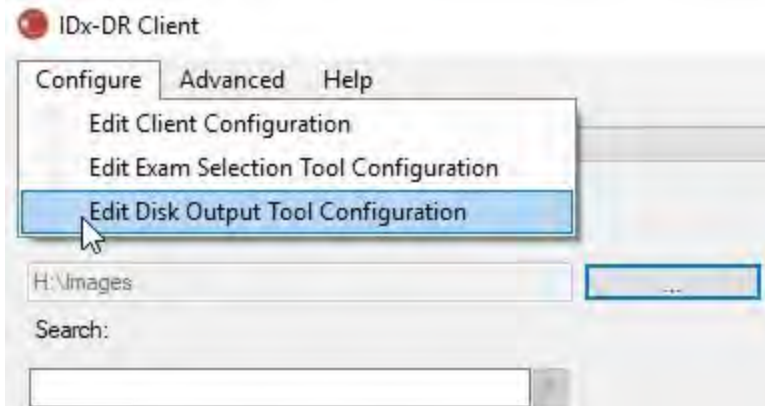
Then define the separator character and specify the filename fields. Using the arrows described in section 4.5 item 3, the order of the active fields can be changed. The minimum elements required are Patient ID and IDx Submission ID. Additional fixed text can be added to the beginning of the filename in the “Prepend:” input box. For example: if “Site 1” was included in the Prepend input box, the filename would be “Site 1_Patient ID_IDx Submission ID”. The filename convention, as it corresponds to the “Active Fields” box, is generated in the “Example” section of the Disk Output Tool Configuration window.

NOTE: The available fields are generated based on the filename configured in the Exam Selection Tool Configuration Window.

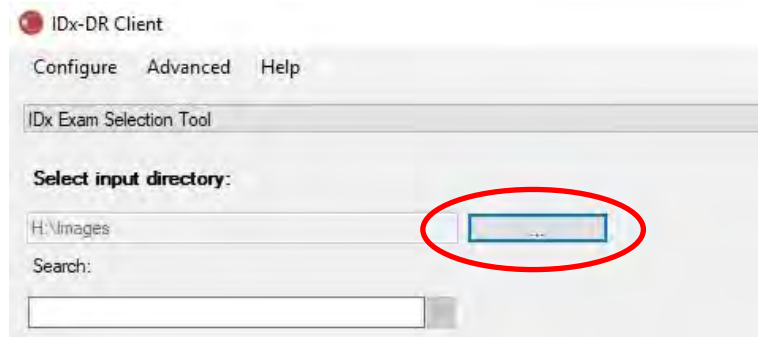
Press the “Save” button to finish configuring the Disk Output Tool.

If the Disk Output Tool Configuration ever needs to be changed, select the “Edit Disk Output Tool Configuration” option under the “Configure” dropdown menu (shown below) and revisit the steps in this section.

4.7 Configure Image Source

After the grouping criterion has been established, the image input directory must be configured. Press the “...” button on the main interface to bring up a directory selection dialog. Select the appropriate directory where patient images are present.

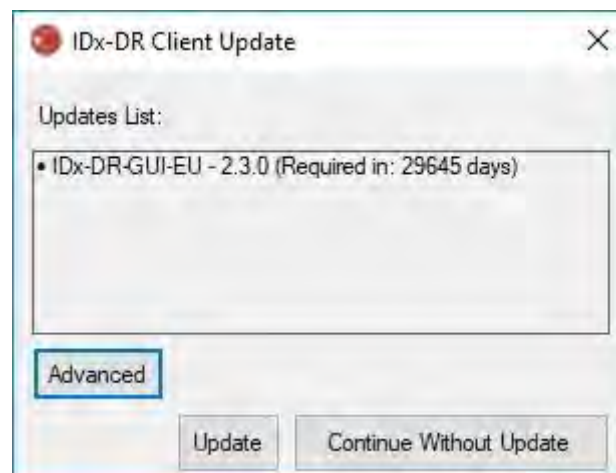
Warning: IDx-DR does not support the selection of a directory with more than 500 images.

If the grouping criterion was configured successfully and there are images of the correct type (see Section 5.1.1) present in the input directory, exams/images should show up in the list.

4.8 Updates

Every time the IDx-DR Client starts, or a set of patient images is submitted, the software checks if an updated version is available. When an updated version of the software has been found, IDx-DR Client will ask whether to download and install the update.

Updates can be optional or required. Optional updates are minor updates that are not immediately required to use the software. Required updates must be installed immediately in order to keep using the software. Optional requirements can become required after a certain time period. This time period is specified in brackets behind the update.

To install the update, press the “Update” button. The update will be downloaded, which can take several minutes depending on the speed of your internet connection. After the download has completed, the installation process will commence automatically. Wait until the installation finishes. The application will restart and will be ready for use.

If an update is still optional select “Continue Without Update” to continue using the software without updating it. If there are less than 3 days (72 hours) until an update becomes required, the software will prompt at every startup and at every exam submission to install the update.

If “Disable all update notifications” is selected (see section 6.1) required updates will not prompt a notification and may result in IDx-DR Client becoming inoperable.

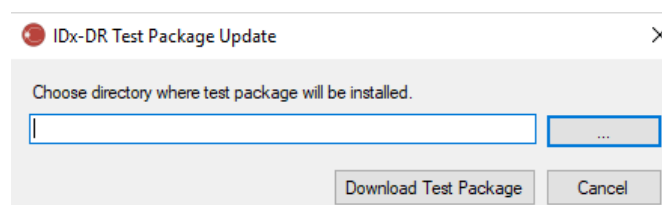
Advanced Update Options, Install Test Version

IDx does extensive testing on all updates but wants to support customers that wish to do their own validation/testing of the client software. This functionality can be used to do testing to verify that an update does not negatively impact your production environment.

Warning: Advanced update functionality is only meant to be used by individuals familiar with the IT systems on which the IDx-DR Client is running.

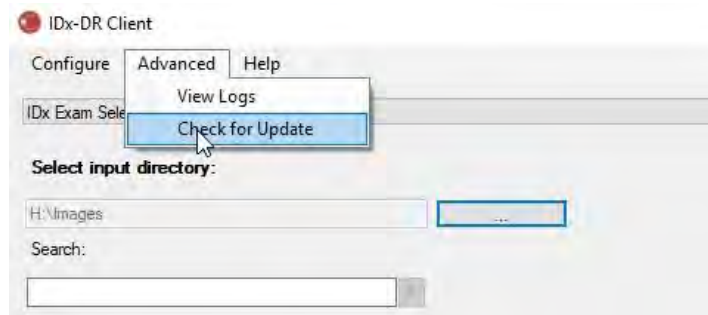
To test the updated software before using it in a clinical environment, click the “Advanced” button in the IDx-DR Client Update window (see previous section). This will show the “IDx-DR Test Package Download” function.

Select a writable directory on the local computer hard-disk using the “...” button and click “Download Test Package”. A sub-directory, IDx-DR_TestPackage, is created in the directory that was indicated in the dialog. The test package directory contains the test installation. When the test software is started, a test license code will have to be entered. Contact IDx to obtain such a code for testing purposes. The testing license will make the client software fully functional for a limited number of test submissions.

Manually Checking for an Update

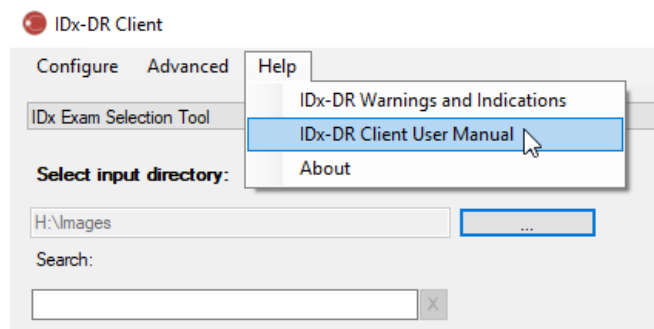
It is also possible to manually check for an update by selecting the “Check for Update” function in the “Advanced” menu:

If an update is available, the regular update screen as described at the start of Section 0 will be shown.

4.9 Accessing the manual and viewing the product label

IDx-DR comes with an electronic manual that is accessed through the “Help” menu.

To open the manual, select “IDx-DR Client User Manual”. The program will open the manual in a separate viewer window.

IDx-DR has an official product label that can also be viewed from the “Help” menu. To view the label as well as information about the version of the software you are using select the “About” option from the menu.

5 Standard Use

5.1 Submitting Retinal Images for Interpretation

If you have not yet registered your copy of the IDx-DR Client, you must do so before you are able to submit exams to IDx-DR. See Section 4 for details about installation and registration of IDx-DR.

Submission Criteria

To obtain an IDx-DR result for a set of images from a patient, the following prerequisites must be met:

1. IDx-DR Client is operational
2. The IDx-DR Service is available
3. You have an active IDx-DR account with IDx
4. There must be a disc centered and a macula centered image for each eye for a total of four images.
5. All images obtained from the same patient on the same day with the same camera
6. The image file format must be JPEG, BMP, TIFF, DICOM (containing JPEG) or PNG

The IDx-DR Client must be connected to IDx servers before submitting any exams. This can be accomplished by examining the connection status label in the bottom right corner of the IDx-DR Client main window. When a connection is available, “Service Available” will appear. When no connection is available “Service Unavailable” will appear.

Selecting and Submitting patient images to IDx-DR using the Exam Selection Tool

All exams from a given folder will be present in the Exam Selection Tool. If an exam is incomplete (has less than 4 images) it will be present in the list but will be marked in red and all individual image files in the exam will be displayed below the Patient ID.

Search:

| Patient ID | File Name | Date |
|----------------|--------------|------------------------|
| - PATIENTID1 | | 2007-09-30 |
| | PATIENTID1_4 | 2007-09-30 7:53:23 PM |
| | PATIENTID1_3 | 2007-09-30 7:53:13 PM |
| | PATIENTID1_2 | 2007-09-30 7:53:03 PM |
| [-] PATIENTID2 | | 2017-02-16 |
| [-] PATIENTID3 | | 2007-09-26 |
| | PATIENTID3_5 | 2007-09-26 12:54:54 PM |
| | PATIENTID3_4 | 2007-09-26 11:54:54 AM |
| | PATIENTID3_3 | 2007-09-26 11:54:45 AM |
| | PATIENTID3_2 | 2007-09-26 11:54:35 AM |
| | PATIENTID3_1 | 2007-09-26 11:54:26 AM |
| [-] PATIENTID4 | | 2007-09-26 |
| [-] PATIENTID5 | | 2014-08-20 |
| [-] PATIENTID6 | | 2014-08-20 |
| [-] PATIENTID7 | | 2015-07-11 |

As soon as an exam is complete, it will be shown as a single line represented by Patient ID and the exam will be ready for submission. Select an exam or individual images by clicking the left mouse button.

Press the “Process Selected Exam” button to submit the complete four image exam.

If an exam contains more than four images, all individual image files of the exam will be shown below the Patient ID. Manually select the four images to be submitted as part of an exam by clicking on the image names.

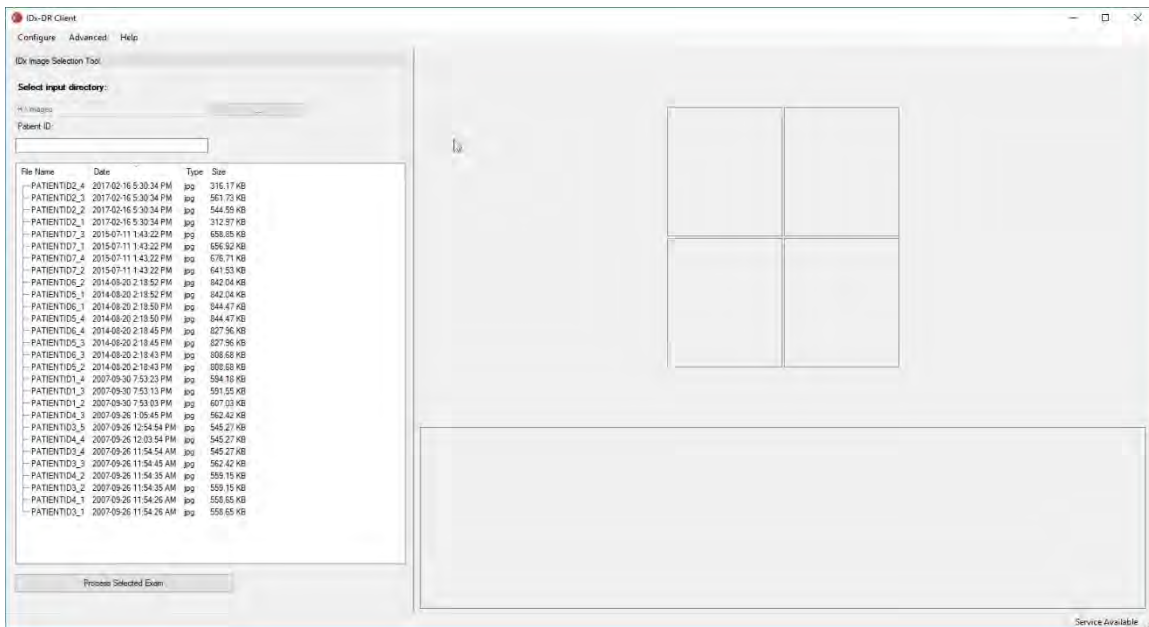
Operators can search for an exam by Patient ID in the bar above the Exam Selection Table. A dropdown box will show Patient IDs that match the search and the associated date(s) of the exam(s). Selecting a Patient ID search result will move to that location in the Exam Selection Table.

Selecting and Submitting patient images to IDx-DR using the Image Selection Tool

The IDx-DR Client provides the Image Selection Tool, which allows operators to manually group images into an exam.

NOTE: The IDx-DR Client can automatically group images into exams by filename or by using information from a DICOM file. For more information, see Section 5.1.2.

After opening the IDx-DR Client with the Image Selection Tool configured, you will see the following main window:

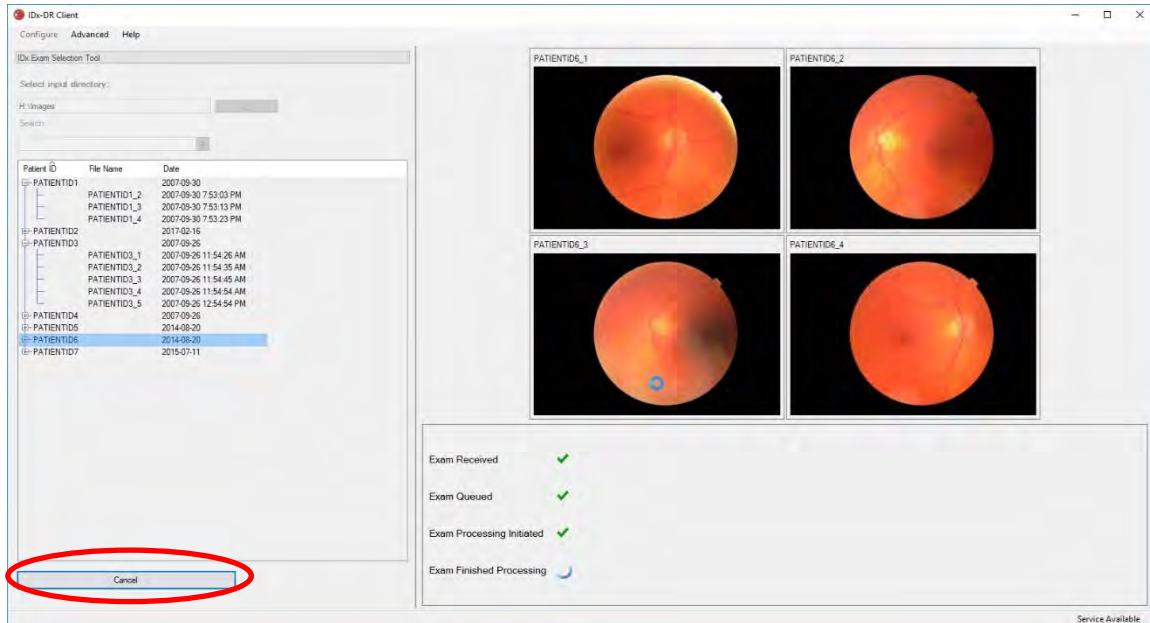
The left side of the main window contains the Image Selection Panel, which lets operators select supported image types from any hard drive location on the computer.

To use the Image Selection Tool:

1. Verify that the software is connected to IDx Service before submitting any exams. This can be accomplished by examining the connection status label in the bottom right corner of the IDx-DR Client main window. When a connection is available, this will say “Service Available.” When no connection is available, it will say “Service Unavailable.”
2. Prepare an exam for submission to IDx-DR by selecting 4 images from the list using the left mouse button. Click once on a filename to select an image. If a selected file is clicked again, the image is de-selected. Every file that is selected in the interface displays a thumbnail. The thumbnails can be used to verify that the correct images were selected and that the correct imaging protocol has been followed. Once four images have been selected the exam is ready for submission.
3. (Optional) To facilitate patient tracking, the IDx-DR Client offers functionality to enter a patient ID that will be linked to the results. Use the Patient text box to enter the patient ID. If no patient ID is entered, the IDx-DR Client will create one that consists of the first image filename.
4. Press the “Process Selected Exam” button to submit the complete four image exam.

- After pressing “Process Selected Exam” it is possible to cancel the exam submission by pressing the cancel button.

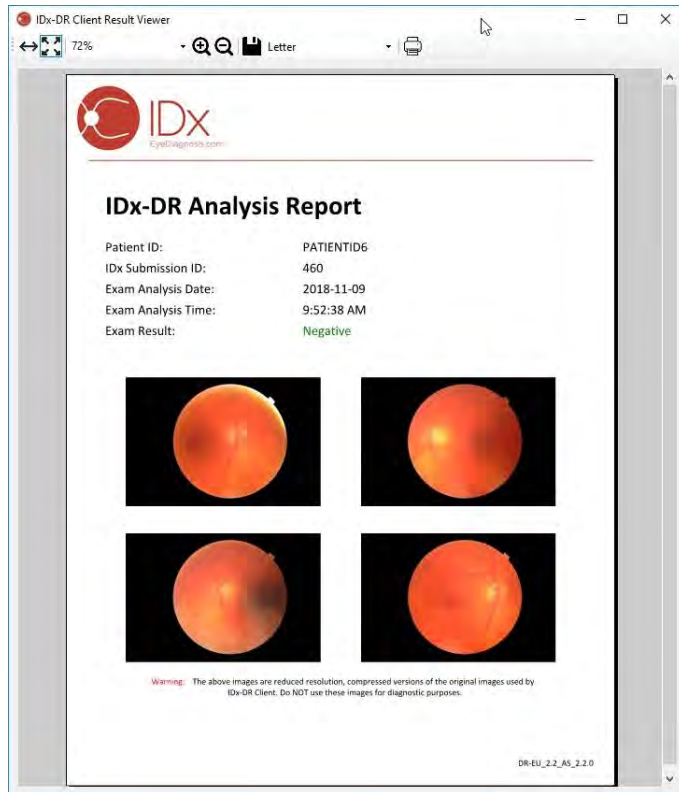
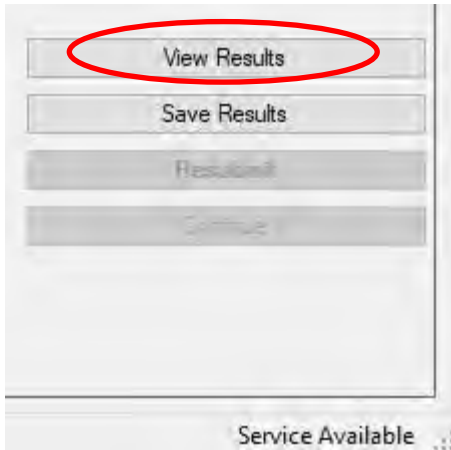
Receiving the Exam Results

During processing, the Exam Status will change from “Exam Received”, “Exam Queued”, “Exam Processing Initiated” to the final status of “Exam Finished Processing”. Once processing has finished, the application displays the result and makes the analysis report available.

The Analysis Report

When processing has finished, the analysis report can be opened using the “View Results” button.

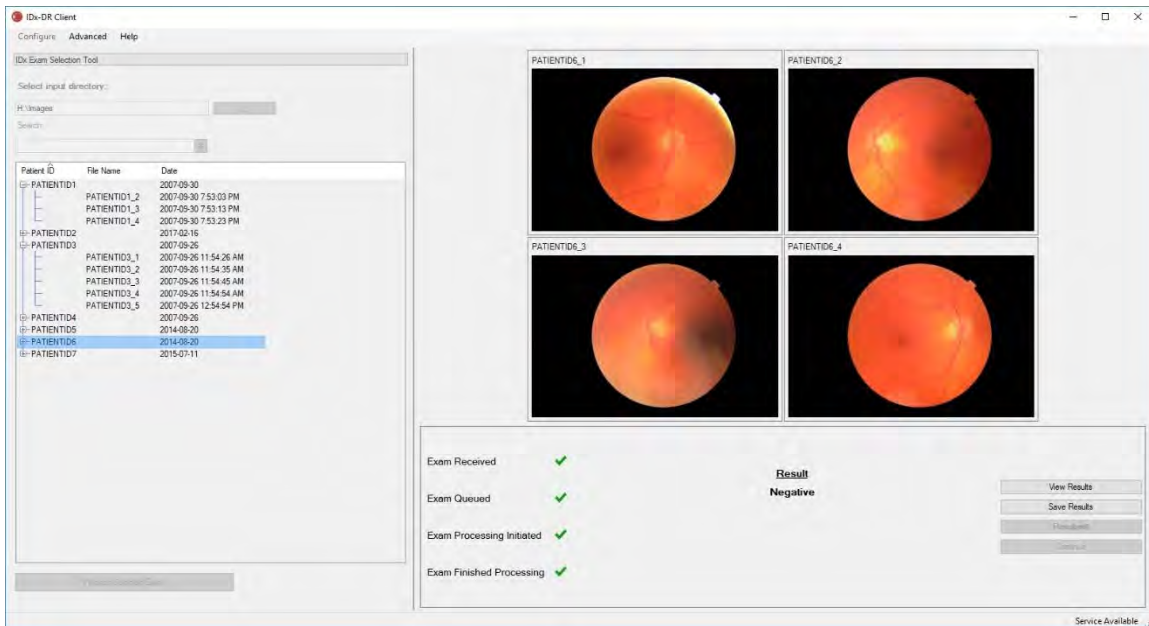
The IDx-DR analysis result is shown in the report. The report viewer allows printing, saving to an output file, and zooming in and out. The output file format can be set using the dropdown tab next to the printer button.

The report itself contains the patient ID, the IDx Submission ID, the date and time of analysis, the version of the IDx-DR software and the screening result. See IDx-DR Warnings and Indications manual for more information on IDx-DR results and referral recommendations.

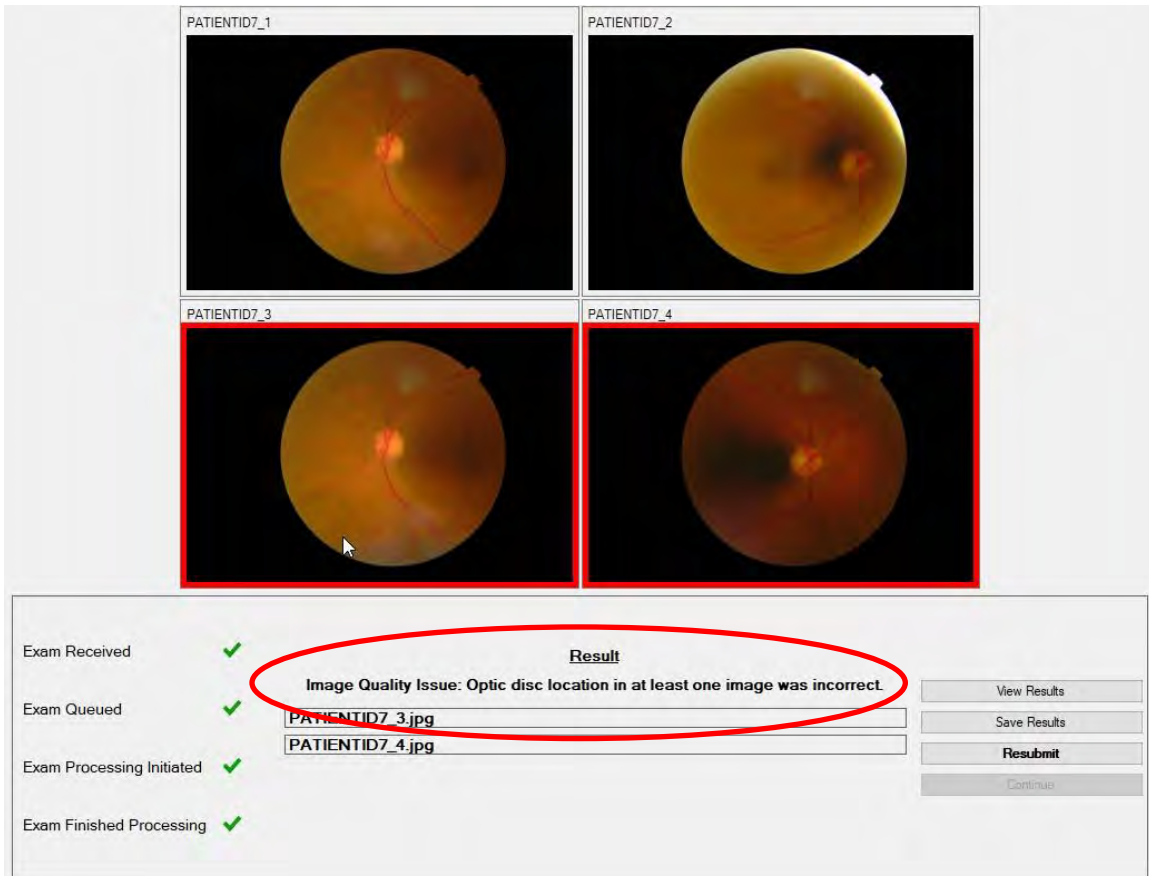
Main interface result display

After processing, the IDx-DR interface appears as below:

When all four images are of sufficient quality, the user can choose to view or save the report. Processing cannot continue until at least one of these options is chosen.

If IDx-DR detects problems with the image or exam quality, the output will be similar to what is below:

| | |
|--------------|--------------|
| PATIENTID7_1 | PATIENTID7_2 |
| PATIENTID7_3 | PATIENTID7_4 |

| | | | |
|---------------------------|---|---|--------------|
| Exam Received | ✓ | <p>Result</p> <p>Image Quality Issue: Optic disc location in at least one image was incorrect.</p> <p>PATIENTID7_3.jpg</p> <p>PATIENTID7_4.jpg</p> | View Results |
| Exam Queued | ✓ | | Save Results |
| Exam Processing Initiated | ✓ | | Resubmit |
| Exam Finished Processing | ✓ | | Continue |

Any images that caused the quality of the exam to be insufficient are marked with a red border (see image above). The result is shown textually in the result box (see red circle in image above).

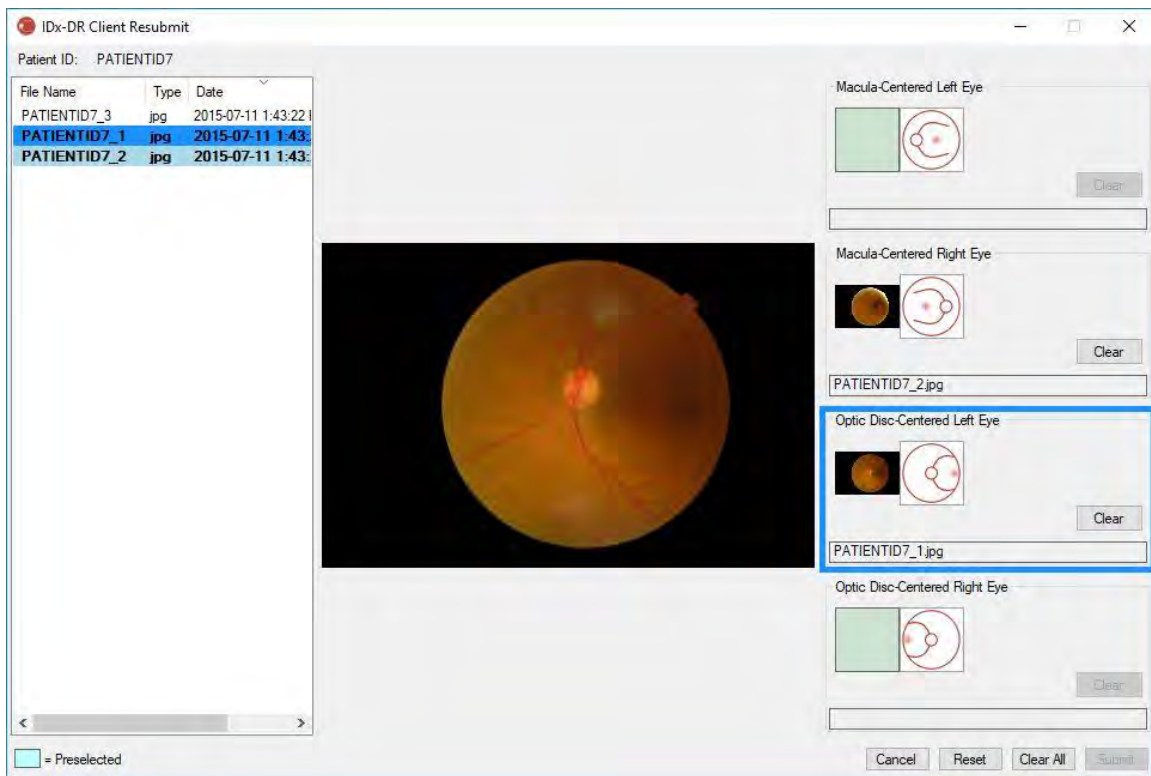
Following the return of a result, the four buttons in the bottom right portion of the Client can be used as follows:

- “View Results” – View the IDx-DR report.
- “Save Results” – Save the IDx-DR report to a file.
- “Resubmit” – Resubmit the same exam after replacing the low quality images.
- “Continue” – This button can only be clicked after viewing or saving the IDx-DR report. Continue will unselect any images and allow the submission of the next exam.

Resubmitting an Exam

To resubmit an exam in which there was an image quality or protocol issue, press the “Resubmit” button in the bottom right portion of the screen after a result is returned. After pressing this button, the resubmission screen (below) is shown:

Only sufficient quality images are retained in the resubmit screen. The software will attempt to place the sufficient images in the correct protocol box (see the figure above). If an image is assigned to the wrong protocol box, the “Clear” button can be used to clear the field.

The image file list, which displays all sufficient quality images for the current patient, is located on the left side of the window. Previously submitted images of insufficient quality are not included in this list. Images that are already assigned to a protocol box have a bold font. The list will automatically refresh when new images are added.

The right side of the window displays the four image protocol categories (left/right, macula/disc centered) required for resubmission. To complete image selection for resubmission, click on the file name of the desired image in the left panel. The image from the selected file will be displayed in the center of the screen for confirmation. You may then either click on the appropriate protocol box or drag and drop the file name in

the box. Once all four images are selected, click the “Submit” button in the bottom right corner of the resubmit window.

The “Reset” button can be used to reassign the preselected images to the corresponding protocol category.

The resubmission process can be repeated multiple times for the same patient.

5.2 Command Line Submission

Interface

The IDx-DR Client offers a command line interface for exam processing. The IDx-DR Client must be registered and configured with the GUI prior to using the command line interface. To initiate a command line submission, call the IDx-DR Client executable with multiple command line options. Required command line options are displayed below:

```
IDxDRClient.exe -L <uuid> <patient identifier> <image 0> <image 1> <image 2> <image 3> <output directory>
```

where the variables represent the following:

- uuid: unique submission id specified by user
- patient identifier: unique patient identifier specified by user
- image 0-3: full path to images for submission
- output directory: the output directory to put the output file(s)

Exam results are automatically saved to the output directory. One of two types of files may be written to the output directory: 1) an analysis report or 2) an error file.

Outputs and Troubleshooting

The presence of an analysis report or an error file indicates processing completion. If no analysis report nor error file appear in the output directory after a period of more than 10 minutes, then processing has likely failed due to another reason, proceed to Section 5.2.2.3 for how to proceed.

Analysis Report

For a description of the analysis report, see Section 5.1.5. The file format of the analysis report may be configured using the IDx-Client GUI by selecting an “Extension Type” from the Disk Output Tool Configuration, see Section 4.6. The analysis report is written to the output directory with the format configured in the Disk Output Tool Configuration.

Error File

If processing fails for any reason, such that an analysis report cannot be created, then an error file is written to the output directory. The error file has the filename <uuid>_ERR.txt (i.e. UUID001_ERR.txt if uuid="UUID001"). The error file will contain:

1. A line describing the error that has occurred, translated to the selected language (see "Language" in Section 0).
2. A troubleshooting code, see Section 7.

Advanced Troubleshooting

There are a few instances where an error file is unable to be written. In these cases, errors are written to the Windows Event Log. The logs are written to the Application Event Log (Source: IDxServices). The logs are written in English, as the translation may be compromised, but will contain an error code defined in the table below. If contacting IDx Customer Support, please have the complete Windows Event Log with the error code available.

| Troubleshooting Code | Likely Cause | Action |
|----------------------|---|--|
| EC: 495P6XD | Incorrect number of command line arguments. | <ol style="list-style-type: none"> 1) Review command line inputs if possible and verify against description in Section 0. 2) If above does not resolve issue, contact IDx Customer Support. |
| EC: OILYLIQ | Output directory specified via command line does not exist or is not writeable. | <ol style="list-style-type: none"> 1) Ensure output directory exists. 2) Ensure user performing submission has write permissions to the output directory. 3) If above does not resolve issue, contact IDx Customer Support. |
| EC: WPQ0ZF5 | Translation file error due to corrupt installation or unsupported language. | <ol style="list-style-type: none"> 1) Start the GUI to determine if the installation is corrupt. If so, you will see an update window similar to Section 4.8 indicating corrupt |

| | | |
|--|--|---|
| | | installation with a “Repair” option. 2) If above does not indicate corrupt install or does not resolve issue, contact IDx Customer Support |
|--|--|---|

Service and Maintenance

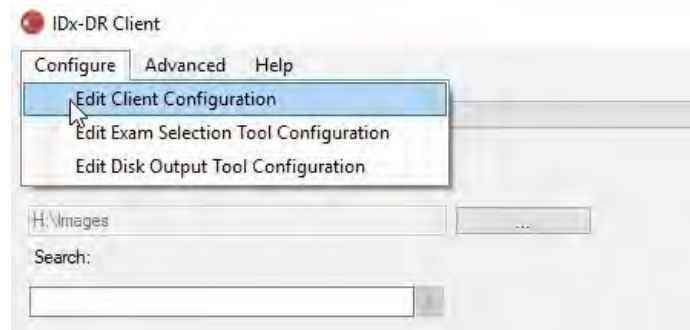
The IDx-DR Client is not independently user serviceable. When service is needed, IDx will release an updated version of the IDx-DR Client that will be automatically downloaded and installed. See Section 0 for information on updates.

6 Advanced Options

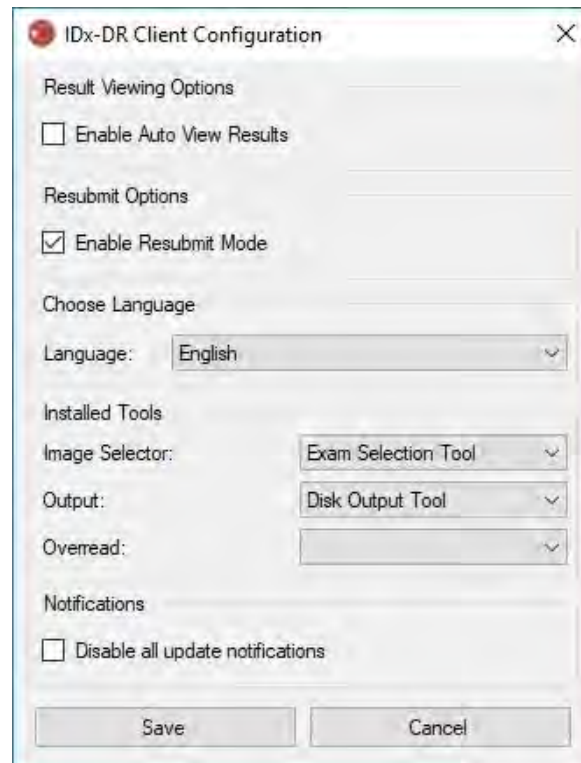
The following functionality is provided for advanced users who need to configure the IDx-DR Client. This functionality is not needed to use the device.

6.1 Advanced configuration

In the top left corner of the IDx-DR Client screen, a menu with configuration options is available:

After selecting “Edit Client Configuration” from the “Configure” tab, the window with configuration options below is shown.

In the previously shown configuration window, the following configuration options are available (from top to bottom):

- “Enable Auto View Results” – When this box is checked, the IDx-DR report will be automatically shown after an analysis if image quality is sufficient.
- “Enable Resubmit Mode” – When this box is checked, the software will prompt to resubmit upon an insufficient image quality result.
- “Choose Language” – If available, different user interface languages can be selected.
- “Image Selector” – A list of input tools that are installed in the IDx-DR Client application.
- “Output ” – A list of output tools that are installed in the IDx-DR Client application to handle exam analysis reports.
- “Notifications” – When this box is checked, IDx-DR Client will disable all update notifications. Updates must be manually checked. See section 4.8.2 for information on manual updates. If “Disable all update notifications” is selected, required updates will not prompt a notification and may result in IDx-DR Client becoming inoperable.

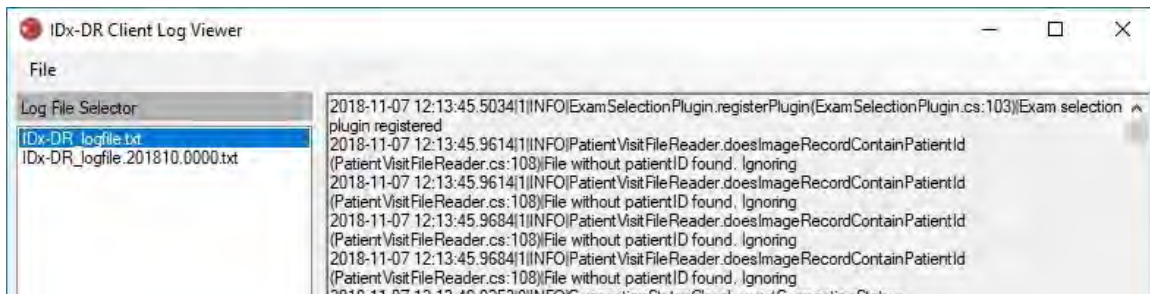
After making a change in the preferences, select the “Save” button to save changes.

6.2 Viewing IDx-DR Client logging information

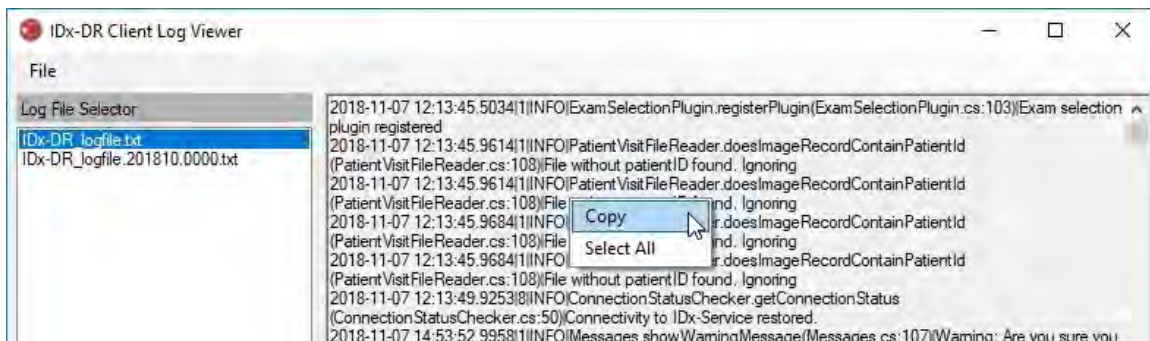
For technical troubleshooting, IDx-DR Client includes a log file viewer. It is selected via the “Advanced” menu:

After selecting the “View Logs” menu option, the following window presents the log file. A specific log file can be viewed by selecting the name of the file on the left side of the window.

The contents of the log file are shown in the right part of the window. Excerpts from the log file can be copied to the clipboard for pasting into another document. To do this, select the text to be copied, and click the right mouse button. Select the “Copy” option from the menu that opens:

The same menu can also be used to select all text before copying. Standard keyboard shortcuts, including Ctrl+C to Copy and Ctrl+A to Select All, will also work.

7 IDx-DR Troubleshooting

It is recommended that an IT professional handle troubleshooting.

The following table defines troubleshooting codes that may be found in IDx-DR Client error messages. If the troubleshooting steps below do not resolve the problem, please contact IDx at +1 (319) 248-5620 or support@eyediagnosis.net.

| Troubleshooting Code | Likely Cause | Action |
|----------------------|---|--|
| 001 | No customer resolution. | Contact IDx Customer Support. |
| 101 | Customer internet connection is down. | <ol style="list-style-type: none"> 1) Check the computer internet connection. 2) Ensure computer firewall is not blocking IDx-DR communication through port 443. |
| 102 | IDx servers are down. | <ol style="list-style-type: none"> 1) Check local internet connection according to Troubleshooting Code 101 above. 2) Check image file size. 3) Wait and attempt later. |
| 103 | Patient image(s) deleted or corrupted | Check image input directory to ensure patient images exist and are not corrupted |
| 201 | Read/write permissions of install folder. | <ol style="list-style-type: none"> 1) Navigate to the install location of IDx-DR. 2) Verify read/write permissions on install directory and corresponding child directories/files. 3) Verify sufficient disk space on hard drive/partition where IDx-DR is installed. |
| 202 | Read/write permissions of the report save folder. | Check permissions on the selected manual and auto save directories. |
| 301 | Update file corruption. | Attempt downloading update again. |
| 302 | File Corruption. | Restart IDx-DR Client and attempt update. |

8 Reference Information

Computer requirements

| | |
|-------------------------|---|
| Hardware | Intel architecture |
| Minimum processor | Pentium 4 |
| Minimum internal memory | 2 Gigabytes |
| Minimum disk space | 250MB |
| Operating system | Microsoft: Windows 7 32- or 64-bit Windows 8.1 Update 32- or 64-bit Windows 10 32- or 64-bit |

9 Software Licensing Information

The following third-party software libraries were licensed for the development of IDx-DR Client.

Nlog

Copyright (c) 2004-2017 Jaroslaw Kowalski <jaak@jkwalski.net>, Kim Christensen, Julian Verdurmen All rights reserved. Redistribution and use in source and binary forms, with or without modification, are permitted provided that the following conditions are met: * Redistributions of source code must retain the above copyright notice, this list of conditions and the following disclaimer. * Redistributions in binary form must reproduce the above copyright notice, this list of conditions and the following disclaimer in the documentation and/or other materials provided with the distribution. * Neither the name of Jaroslaw Kowalski nor the names of its contributors may be used to endorse or promote products derived from this software without specific prior written permission. THIS SOFTWARE IS PROVIDED BY THE COPYRIGHT HOLDERS AND CONTRIBUTORS "AS IS" AND ANY EXPRESS OR IMPLIED WARRANTIES, INCLUDING, BUT NOT LIMITED TO, THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE ARE DISCLAIMED. IN NO EVENT SHALL THE COPYRIGHT OWNER OR CONTRIBUTORS BE LIABLE FOR ANY DIRECT, INDIRECT, INCIDENTAL, SPECIAL, EXEMPLARY, OR CONSEQUENTIAL DAMAGES (INCLUDING, BUT NOT LIMITED TO, PROCUREMENT OF SUBSTITUTE GOODS OR SERVICES; LOSS OF USE, DATA, OR PROFITS; OR BUSINESS INTERRUPTION) HOWEVER CAUSED AND ON ANY THEORY OF LIABILITY, WHETHER IN CONTRACT, STRICT LIABILITY, OR TORT (INCLUDING NEGLIGENCE OR OTHERWISE) ARISING IN ANY WAY OUT OF THE USE OF THIS SOFTWARE, EVEN IF ADVISED OF THE POSSIBILITY OF SUCH DAMAGE.

NSubstitute

Copyright (c) 2009 Anthony Egerton (nsubstitute@delfish.com) and David Tchepak (dave@davesquared.net) All rights reserved. Redistribution and use in source and binary forms, with or without modification, are permitted provided that the following conditions are met: * Redistributions of source code must retain the above copyright notice, this list of conditions and the following disclaimer. * Redistributions in binary form must reproduce the above copyright notice, this list of conditions and the following disclaimer in the documentation and/or other materials provided with the distribution. * Neither the names of the copyright holders nor the names of its contributors may be used to endorse or promote products derived from this software without specific prior written permission. THIS SOFTWARE IS PROVIDED BY THE COPYRIGHT HOLDERS AND CONTRIBUTORS "AS IS" AND ANY EXPRESS OR IMPLIED WARRANTIES, INCLUDING, BUT NOT LIMITED TO, THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE ARE DISCLAIMED. IN NO EVENT SHALL THE COPYRIGHT OWNER OR CONTRIBUTORS BE LIABLE FOR ANY DIRECT, INDIRECT, INCIDENTAL, SPECIAL, EXEMPLARY, OR CONSEQUENTIAL DAMAGES (INCLUDING, BUT NOT LIMITED TO, PROCUREMENT OF SUBSTITUTE GOODS OR SERVICES; LOSS OF USE, DATA, OR PROFITS; OR BUSINESS INTERRUPTION) HOWEVER CAUSED AND ON ANY THEORY OF LIABILITY, WHETHER IN CONTRACT, STRICT LIABILITY, OR TORT (INCLUDING NEGLIGENCE OR OTHERWISE) ARISING IN ANY WAY OUT OF THE USE OF THIS SOFTWARE, EVEN IF ADVISED OF THE POSSIBILITY OF SUCH DAMAGE.

DnsClient.NET and RestSharp

Apache License
Version 2.0, January 2004
<http://www.apache.org/licenses/>
TERMS AND CONDITIONS FOR USE, REPRODUCTION, AND DISTRIBUTION

1. Definitions.

"License" shall mean the terms and conditions for use, reproduction, and distribution as defined by Sections 1 through 9 of this document.

"Licensor" shall mean the copyright owner or entity authorized by the copyright owner that is granting the License.

"Legal Entity" shall mean the union of the acting entity and all other entities that control, are controlled by, or are under common control with that entity. For the purposes of this definition, "control" means (i) the power, direct or indirect, to cause the direction or management of such entity, whether by contract or otherwise, or (ii) ownership of fifty percent (50%) or more of the outstanding shares, or (iii) beneficial ownership of such entity.

"You" (or "Your") shall mean an individual or Legal Entity exercising permissions granted by this License.

"Source" form shall mean the preferred form for making modifications, including but not limited to software source code, documentation source, and configuration files.

"Object" form shall mean any form resulting from mechanical transformation or translation of a Source form, including but not limited to compiled object code, generated documentation, and conversions to other media types.

"Work" shall mean the work of authorship, whether in Source or Object form, made available under the License, as indicated by a copyright notice that is included in or attached to the work (an example is provided in the Appendix below).

"Derivative Works" shall mean any work, whether in Source or Object form, that is based on (or derived from) the Work and for which the editorial revisions, annotations, elaborations, or other modifications represent, as a whole, an original work of authorship. For the purposes of this License, Derivative Works shall not include works that remain separable from, or merely link (or bind by name) to the interfaces of, the Work and Derivative Works thereof.

"Contribution" shall mean any work of authorship, including the original version of the Work and any modifications or additions to that Work or Derivative Works thereof, that is intentionally submitted to Licensor for inclusion in the Work by the copyright owner or by an individual or Legal Entity authorized to submit on behalf of the copyright owner. For the purposes of this definition, "submitted" means any form of electronic, verbal, or written communication sent to the Licensor or its representatives, including but not limited to communication on electronic mailing lists, source code control systems, source code control systems, and issue tracking systems that are managed by, or on behalf of, the Licensor for the purpose of discussing and improving the Work, but excluding communication that is conspicuously marked or otherwise designated in writing by the copyright owner as "Not a Contribution."

"Contributor" shall mean Licensor and any individual or Legal Entity on behalf of whom a Contribution has been received by Licensor and subsequently incorporated within the Work.

2. Grant of Copyright License. Subject to the terms and conditions of this License, each Contributor hereby grants to You a perpetual, worldwide, non-exclusive, no-charge, royalty-free, irrevocable copyright license to reproduce, prepare Derivative Works of, publicly display, publicly perform, sublicense, and distribute the Work and such Derivative Works in Source or Object form.

3. Grant of Patent License. Subject to the terms and conditions of this License, each Contributor hereby grants to You a perpetual, worldwide, non-exclusive, no-charge, royalty-free, irrevocable (except as stated in this section) patent license to make, have made, use, offer to sell, sell, import, and otherwise transfer the Work, where such license applies only to those patent claims licensable by such Contributor that are necessarily infringed by their Contribution(s) alone or by combination of their Contribution(s) with the Work to which such Contribution(s) was submitted. If You institute patent litigation against any entity (including a cross-claim or counterclaim in a lawsuit) alleging that the Work or a Contribution incorporated within the Work constitutes direct or contributory patent infringement, then any patent licenses granted to You under this License for that Work shall terminate as of the date such litigation is filed.

4. Redistribution. You may reproduce and distribute copies of the Work or Derivative Works thereof in any medium, with or without modifications, and in Source or Object form, provided that You meet the following conditions:

You must give any other recipients of the Work or Derivative Works a copy of this License; and

You must cause any modified files to carry prominent notices stating that You changed the files; and

You must retain, in the Source form of any Derivative Works that You distribute, all copyright, patent, trademark, and attribution notices from the Source form of the Work, excluding those notices that do not pertain to any part of the Derivative Works; and

If the Work includes a "NOTICE" text file as part of its distribution, then any Derivative Works that You distribute must include a readable copy of the attribution notices contained within such NOTICE file, excluding those notices that do not pertain to any part of the Derivative Works, in at least one of the following places: within a NOTICE text file distributed as part of the Derivative Works; within the Source form of documentation, if provided along with the Derivative Works; or, within a display generated by the Derivative Works, if and wherever such third-party notices normally appear. The contents of the NOTICE file are for informational purposes only and do not modify the License. You may add Your own attribution notices within Derivative Works that You distribute, alongside or as an addendum to the NOTICE text from the Work, provided that such additional attribution notices cannot be construed as modifying the License.

You may add Your own copyright statement to Your modifications and may provide additional or different license terms and conditions for use, reproduction, or distribution of Your modifications, or for any such Derivative Works as a whole, provided Your use, reproduction, and distribution of the Work otherwise complies with the conditions stated in this License.

5. Submission of Contributions. Unless You explicitly state otherwise, any Contribution intentionally submitted for inclusion in the Work by You to the Licensor shall be under the terms and conditions of this License, without any additional terms or conditions. Notwithstanding the above, nothing herein shall supersede or modify the terms of any separate license agreement you may have executed with Licensor regarding such Contributions.

6. Trademarks. This License does not grant permission to use the trade names, trademarks, service marks, or product names of the Licensor, except as required for reasonable and customary use in describing the origin of the Work and reproducing the content of the NOTICE file.

7. Disclaimer of Warranty. Unless required by applicable law or agreed to in writing, Licensor provides the Work (and each Contributor provides its Contributions) on an "AS IS" BASIS, WITHOUT WARRANTIES OR CONDITIONS OF ANY KIND, either express or implied, including, without limitation, any warranties or conditions of TITLE, NON-INFRINGEMENT, MERCHANTABILITY, or FITNESS FOR A PARTICULAR PURPOSE. You are solely responsible for determining the appropriateness of using or redistributing the Work and assume any risks associated with Your exercise of permissions under this License.

8. Limitation of Liability. In no event and under no legal theory, whether in tort (including negligence), contract, or otherwise, unless required by applicable law (such as deliberate and grossly negligent acts) or agreed to in writing, shall any Contributor be liable to You for damages, including any direct, indirect, special, incidental, or consequential damages of any character arising as a result of this License or out of the use or inability to use the Work (including but not limited to damages for loss of goodwill, work stoppage, computer failure or malfunction, or any and all other commercial damages or losses), even if such Contributor has been advised of the possibility of such damages.

9. Accepting Warranty or Additional Liability. While redistributing the Work or Derivative Works thereof, You may choose to offer, and charge a fee for, acceptance of support, warranty, indemnity, or other liability obligations and/or rights consistent with this License. However, in accepting such obligations, You may act only on Your own behalf and on Your sole responsibility, not on behalf of any other Contributor, and only if You agree to indemnify, defend, and hold each Contributor harmless for any liability incurred by, or claims asserted against, such Contributor by reason of your accepting any such warranty or additional liability.

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Nlog

Copyright (c) 2004-2017 Jaroslav Kowalski <jaak@kowalski.net>, Kim Christensen, Julian Verdurmen All rights reserved. Redistribution and use in source and binary forms, with or without modification, are permitted provided that the following conditions are met: * Redistributions of source code must retain the above copyright notice, this list of conditions and the following disclaimer. * Redistributions in binary form must reproduce the above copyright notice, this list of conditions and the following disclaimer in the documentation and/or other materials provided with the distribution. * Neither the name of Jaroslav Kowalski nor the names of its contributors may be used to endorse or promote products derived from this software without specific prior written permission. THIS SOFTWARE IS PROVIDED BY THE COPYRIGHT HOLDERS AND CONTRIBUTORS "AS IS" AND ANY EXPRESS OR IMPLIED WARRANTIES, INCLUDING, BUT NOT LIMITED TO, THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE ARE DISCLAIMED. IN NO EVENT SHALL THE COPYRIGHT OWNER OR CONTRIBUTORS BE LIABLE FOR ANY DIRECT, INDIRECT, INCIDENTAL, SPECIAL, EXEMPLARY, OR CONSEQUENTIAL DAMAGES (INCLUDING, BUT NOT LIMITED TO, PROCUREMENT OF SUBSTITUTE GOODS OR SERVICES; LOSS OF USE, DATA, OR PROFITS; OR BUSINESS INTERRUPTION) HOWEVER CAUSED AND ON ANY THEORY OF LIABILITY, WHETHER IN CONTRACT, STRICT LIABILITY, OR TORT (INCLUDING NEGLIGENCE OR OTHERWISE) ARISING IN ANY WAY OUT OF THE USE OF THIS SOFTWARE, EVEN IF ADVISED OF THE POSSIBILITY OF SUCH DAMAGE.

NSubstitute

Copyright (c) 2009 Anthony Egerton (nsubstitute@delfish.com) and David Tchepek (dave@davesquared.net) All rights reserved. Redistribution and use in source and binary forms, with or without modification, are permitted provided that the following conditions are met: * Redistributions of source code must retain the above copyright notice, this list of conditions and the following disclaimer. * Redistributions in binary form must reproduce the above copyright notice, this list of conditions and the following disclaimer in the documentation and/or other materials provided with the distribution. * Neither the names of the copyright holders nor the names of contributors may be used to endorse or promote products derived from this software without specific prior written permission. THIS SOFTWARE IS PROVIDED BY THE COPYRIGHT HOLDERS AND CONTRIBUTORS "AS IS" AND ANY EXPRESS OR IMPLIED WARRANTIES, INCLUDING, BUT NOT LIMITED TO, THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE ARE DISCLAIMED. IN NO EVENT SHALL THE COPYRIGHT OWNER OR CONTRIBUTORS BE LIABLE FOR ANY DIRECT, INDIRECT, INCIDENTAL, SPECIAL, EXEMPLARY, OR CONSEQUENTIAL DAMAGES (INCLUDING, BUT NOT LIMITED TO, PROCUREMENT OF SUBSTITUTE GOODS OR SERVICES; LOSS OF USE, DATA, OR PROFITS; OR BUSINESS INTERRUPTION) HOWEVER CAUSED AND ON ANY THEORY OF LIABILITY, WHETHER IN CONTRACT, STRICT LIABILITY, OR TORT (INCLUDING NEGLIGENCE OR OTHERWISE) ARISING IN ANY WAY OUT OF THE USE OF THIS SOFTWARE, EVEN IF ADVISED OF THE POSSIBILITY OF SUCH DAMAGE.

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If the Work includes a "NOTICE" text file as part of its distribution, then any Derivative Works that You distribute must include a readable copy of the attribution notices contained within such NOTICE file, excluding those notices that do not pertain to any part of the Derivative Works, in at least one of the following places: within a NOTICE text file distributed as part of the Derivative Works; within the Source form or documentation, if provided along with the Derivative Works; or, within a display generated by the Derivative Works, if and wherever such third-party notices normally appear. The contents of the NOTICE file are for informational purposes only and do not modify the License. You may add Your own attribution notices within Derivative Works that You distribute, alongside or as an addendum to the NOTICE text from the Work, provided that such additional attribution notices cannot be construed as modifying the License.

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Nlog

Copyright (c) 2004-2017 Jaroslaw Kowalski <jaak@jkwowski.net>, Kim Christensen, Julian Verdumen All rights reserved. Redistribution and use in source and binary forms, with or without modification, are permitted provided that the following conditions are met: * Redistributions of source code must retain the above copyright notice, this list of conditions and the following disclaimer. * Redistributions in binary form must reproduce the above copyright notice, this list of conditions and the following disclaimer in the documentation and/or other materials provided with the distribution. * Neither the name of Jaroslaw Kowalski nor the names of its contributors may be used to endorse or promote products derived from this software without specific prior written permission. THIS SOFTWARE IS PROVIDED BY THE COPYRIGHT HOLDERS AND CONTRIBUTORS "AS IS" AND ANY EXPRESS OR IMPLIED WARRANTIES, INCLUDING, BUT NOT LIMITED TO, THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE ARE DISCLAIMED. IN NO EVENT SHALL THE COPYRIGHT OWNER OR CONTRIBUTORS BE LIABLE FOR ANY DIRECT, INDIRECT, INCIDENTAL, SPECIAL, EXEMPLARY, OR CONSEQUENTIAL DAMAGES (INCLUDING, BUT NOT LIMITED TO, PROCUREMENT OF SUBSTITUTE GOODS OR SERVICES; LOSS OF USE, DATA, OR PROFITS; OR BUSINESS INTERRUPTION) HOWEVER CAUSED AND ON ANY THEORY OF LIABILITY, WHETHER IN CONTRACT, STRICT LIABILITY, OR TORT (INCLUDING NEGLIGENCE OR OTHERWISE) ARISING IN ANY WAY OUT OF THE USE OF THIS SOFTWARE, EVEN IF ADVISED OF THE POSSIBILITY OF SUCH DAMAGE.

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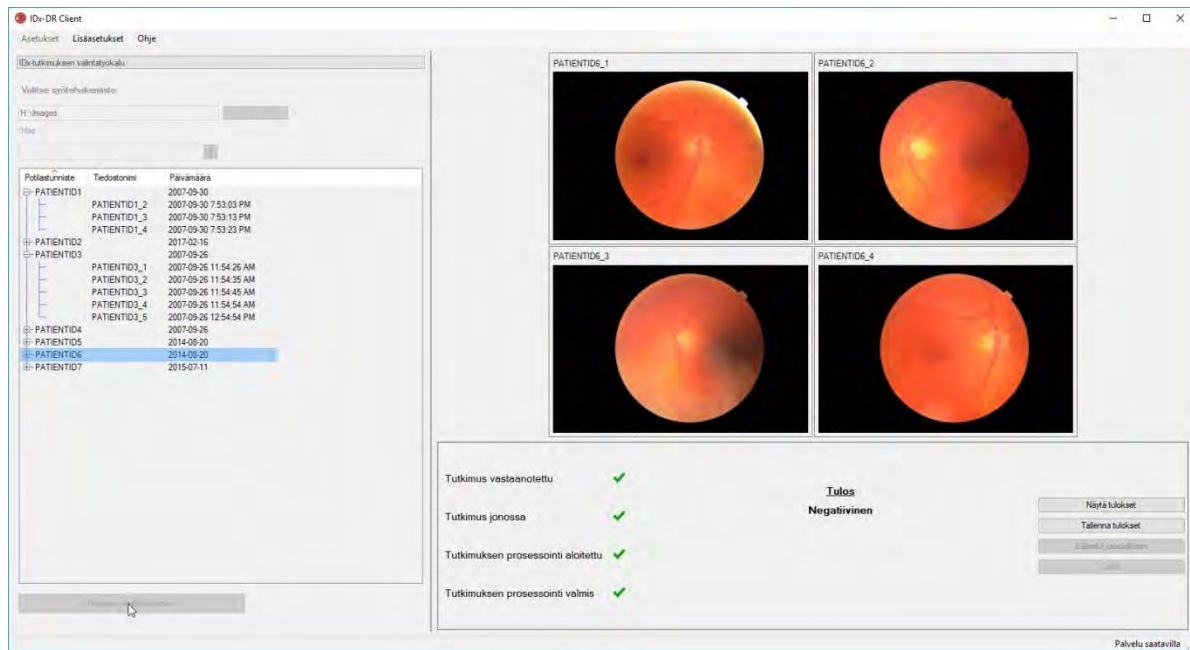
The screenshot shows the IDx-DR Client software interface. On the left, there is a patient list table with columns for Patient ID, Doctor Name, and Date. The patient PATIENTID6 is selected. On the right, there are four fundus images labeled PATIENTID_1 through PATIENTID_4. Below the images, a status section shows 'Tulos Negatiivinen' (Result Negative) and a list of process steps, all marked with green checkmarks. On the far right, there are buttons for 'Näytä tulokset' (Show results), 'Tallenna tulokset' (Save results), and 'Palauta & luo uusia kuvia' (Return & create new images).

| Potilastunnus | Tutkittu nimi | Päivämäärä |
|-------------------|---------------|------------------------|
| PATIENTID1 | | 2007-09-30 |
| PATIENTID1_2 | | 2007-09-30 7:53:03 PM |
| PATIENTID1_3 | | 2007-09-30 7:53:13 PM |
| PATIENTID1_4 | | 2007-09-30 7:53:23 PM |
| PATIENTID2 | | 2017-02-16 |
| PATIENTID3 | | 2007-09-26 |
| PATIENTID3_1 | | 2007-09-26 11:54:26 AM |
| PATIENTID3_2 | | 2007-09-26 11:54:36 AM |
| PATIENTID3_3 | | 2007-09-26 11:54:46 AM |
| PATIENTID3_4 | | 2007-09-26 11:54:54 AM |
| PATIENTID3_5 | | 2007-09-26 12:54:54 PM |
| PATIENTID4 | | 2007-09-26 |
| PATIENTID5 | | 2014-08-20 |
| PATIENTID6 | | 2014-09-20 |
| PATIENTID7 | | 2015-07-11 |

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Nlog

Copyright (c) 2004-2017 Jaroslaw Kowalski <jaak@jkwowski.net>, Kim Christensen, Julian Verdurmen All rights reserved. Redistribution and use in source and binary forms, with or without modification, are permitted provided that the following conditions are met: * Redistributions of source code must retain the above copyright notice, this list of conditions and the following disclaimer. * Redistributions in binary form must reproduce the above copyright notice, this list of conditions and the following disclaimer in the documentation and/or other materials provided with the distribution. * Neither the name of Jaroslaw Kowalski nor the names of its contributors may be used to endorse or promote products derived from this software without specific prior written permission. THIS SOFTWARE IS PROVIDED BY THE COPYRIGHT HOLDERS AND CONTRIBUTORS "AS IS" AND ANY EXPRESS OR IMPLIED WARRANTIES, INCLUDING, BUT NOT LIMITED TO, THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE ARE DISCLAIMED. IN NO EVENT SHALL THE COPYRIGHT OWNER OR CONTRIBUTORS BE LIABLE FOR ANY DIRECT, INDIRECT, INCIDENTAL, SPECIAL, EXEMPLARY, OR CONSEQUENTIAL DAMAGES (INCLUDING, BUT NOT LIMITED TO, PROCUREMENT OF SUBSTITUTE GOODS OR SERVICES; LOSS OF USE, DATA, OR PROFITS; OR BUSINESS INTERRUPTION) HOWEVER CAUSED AND ON ANY THEORY OF LIABILITY, WHETHER IN CONTRACT, STRICT LIABILITY, OR TORT (INCLUDING NEGLIGENCE OR OTHERWISE) ARISING IN ANY WAY OUT OF THE USE OF THIS SOFTWARE, EVEN IF ADVISED OF THE POSSIBILITY OF SUCH DAMAGE.

NSubstitute

Copyright (c) 2009 Anthony Egerton (nsubstitute@delfish.com) and David Tchepak (dave@davesquared.net) All rights reserved. Redistribution and use in source and binary forms, with or without modification, are permitted provided that the following conditions are met: * Redistributions of source code must retain the above copyright notice, this list of conditions and the following disclaimer. * Redistributions in binary form must reproduce the above copyright notice, this list of conditions and the following disclaimer in the documentation and/or other materials provided with the distribution. * Neither the names of the copyright holders nor the names of contributors may be used to endorse or promote products derived from this software without specific prior written permission. THIS SOFTWARE IS PROVIDED BY THE COPYRIGHT HOLDERS AND CONTRIBUTORS "AS IS" AND ANY EXPRESS OR IMPLIED WARRANTIES, INCLUDING, BUT NOT LIMITED TO, THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE ARE DISCLAIMED. IN NO EVENT SHALL THE COPYRIGHT OWNER OR CONTRIBUTORS BE LIABLE FOR ANY DIRECT, INDIRECT, INCIDENTAL, SPECIAL, EXEMPLARY, OR CONSEQUENTIAL DAMAGES (INCLUDING, BUT NOT LIMITED TO, PROCUREMENT OF SUBSTITUTE GOODS OR SERVICES; LOSS OF USE, DATA, OR PROFITS; OR BUSINESS INTERRUPTION) HOWEVER CAUSED AND ON ANY THEORY OF LIABILITY, WHETHER IN CONTRACT, STRICT LIABILITY, OR TORT (INCLUDING NEGLIGENCE OR OTHERWISE) ARISING IN ANY WAY OUT OF THE USE OF THIS SOFTWARE, EVEN IF ADVISED OF THE POSSIBILITY OF SUCH DAMAGE.

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Nlog

Copyright (c) 2004-2017 Jaroslaw Kowalski <jaak@jkwalski.net>, Kim Christensen, Julian Verdumen All rights reserved. Redistribution and use in source and binary forms, with or without modification, are permitted provided that the following conditions are met: * Redistributions of source code must retain the above copyright notice, this list of conditions and the following disclaimer. * Redistributions in binary form must reproduce the above copyright notice, this list of conditions and the following disclaimer in the documentation and/or other materials provided with the distribution. * Neither the name of Jaroslaw Kowalski nor the names of its contributors may be used to endorse or promote products derived from this software without specific prior written permission. THIS SOFTWARE IS PROVIDED BY THE COPYRIGHT HOLDERS AND CONTRIBUTORS "AS IS" AND ANY EXPRESS OR IMPLIED WARRANTIES, INCLUDING, BUT NOT LIMITED TO, THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE ARE DISCLAIMED. IN NO EVENT SHALL THE COPYRIGHT OWNER OR CONTRIBUTORS BE LIABLE FOR ANY DIRECT, INDIRECT, INCIDENTAL, SPECIAL, EXEMPLARY, OR CONSEQUENTIAL DAMAGES (INCLUDING, BUT NOT LIMITED TO, PROCUREMENT OF SUBSTITUTE GOODS OR SERVICES; LOSS OF USE, DATA, OR PROFITS; OR BUSINESS INTERRUPTION) HOWEVER CAUSED AND ON ANY THEORY OF LIABILITY, WHETHER IN CONTRACT, STRICT LIABILITY, OR TORT (INCLUDING NEGLIGENCE OR OTHERWISE) ARISING IN ANY WAY OUT OF THE USE OF THIS SOFTWARE, EVEN IF ADVISED OF THE POSSIBILITY OF SUCH DAMAGE.

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"Contribution" shall mean any work of authorship, including the original version of the Work and any modifications or additions to that Work or Derivative Works thereof, that is intentionally submitted to Licensor for inclusion in the Work by the copyright owner or by an individual or Legal Entity authorized to submit on behalf of the copyright owner. For the purposes of this definition, "submitted" means any form of electronic, verbal, or written communication sent to the Licensor or its representatives, including but not limited to communication on electronic mailing lists, source code control systems, source code control systems that are managed by, or on behalf of, the Licensor for the purpose of discussing and improving the Work, but excluding communication that is conspicuously marked or otherwise designated in writing by the copyright owner as "Not a Contribution."

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Nlog

Copyright (c) 2004-2017 Jaroslaw Kowalski <jaak@kowalski.net>, Kim Christensen, Julian Verdurmen All rights reserved. Redistribution and use in source and binary forms, with or without modification, are permitted provided that the following conditions are met: * Redistributions of source code must retain the above copyright notice, this list of conditions and the following disclaimer. * Redistributions in binary form must reproduce the above copyright notice, this list of conditions and the following disclaimer in the documentation and/or other materials provided with the distribution. * Neither the name of Jaroslaw Kowalski nor the names of its contributors may be used to endorse or promote products derived from this software without specific prior written permission. THIS SOFTWARE IS PROVIDED BY THE COPYRIGHT HOLDERS AND CONTRIBUTORS "AS IS" AND ANY EXPRESS OR IMPLIED WARRANTIES, INCLUDING, BUT NOT LIMITED TO, THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE ARE DISCLAIMED. IN NO EVENT SHALL THE COPYRIGHT OWNER OR CONTRIBUTORS BE LIABLE FOR ANY DIRECT, INDIRECT, INCIDENTAL, SPECIAL, EXEMPLARY, OR CONSEQUENTIAL DAMAGES (INCLUDING, BUT NOT LIMITED TO, PROCUREMENT OF SUBSTITUTE GOODS OR SERVICES; LOSS OF USE, DATA, OR PROFITS; OR BUSINESS INTERRUPTION) HOWEVER CAUSED AND ON ANY THEORY OF LIABILITY, WHETHER IN CONTRACT, STRICT LIABILITY, OR TORT (INCLUDING NEGLIGENCE OR OTHERWISE) ARISING IN ANY WAY OUT OF THE USE OF THIS SOFTWARE, EVEN IF ADVISED OF THE POSSIBILITY OF SUCH DAMAGE.

NSubstitute

Copyright (c) 2009 Anthony Egerton (nsubstitute@defish.com) and David Tchepek (dave@davesquared.net) All rights reserved. Redistribution and use in source and binary forms, with or without modification, are permitted provided that the following conditions are met: * Redistributions of source code must retain the above copyright notice, this list of conditions and the following disclaimer. * Redistributions in binary form must reproduce the above copyright notice, this list of conditions and the following disclaimer in the documentation and/or other materials provided with the distribution. * Neither the names of the copyright holders nor the names of its contributors may be used to endorse or promote products derived from this software without specific prior written permission. THIS SOFTWARE IS PROVIDED BY THE COPYRIGHT HOLDERS AND CONTRIBUTORS "AS IS" AND ANY EXPRESS OR IMPLIED WARRANTIES, INCLUDING, BUT NOT LIMITED TO, THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE ARE DISCLAIMED. IN NO EVENT SHALL THE COPYRIGHT OWNER OR CONTRIBUTORS BE LIABLE FOR ANY DIRECT, INDIRECT, INCIDENTAL, SPECIAL, EXEMPLARY, OR CONSEQUENTIAL DAMAGES (INCLUDING, BUT NOT LIMITED TO, PROCUREMENT OF SUBSTITUTE GOODS OR SERVICES; LOSS OF USE, DATA, OR PROFITS; OR BUSINESS INTERRUPTION) HOWEVER CAUSED AND ON ANY THEORY OF LIABILITY, WHETHER IN CONTRACT, STRICT LIABILITY, OR TORT (INCLUDING NEGLIGENCE OR OTHERWISE) ARISING IN ANY WAY OUT OF THE USE OF THIS SOFTWARE, EVEN IF ADVISED OF THE POSSIBILITY OF SUCH DAMAGE.

DnsClient.NET and RestSharp

Apache License

Version 2.0, January 2004

<http://www.apache.org/licenses/>

TERMS AND CONDITIONS FOR USE, REPRODUCTION, AND DISTRIBUTION

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Nlog

Copyright (c) 2004-2017 Jaroslaw Kowalski <jaak@jkowalski.net>, Kim Christensen, Julian Verdurmen All rights reserved. Redistribution and use in source and binary forms, with or without modification, are permitted provided that the following conditions are met: * Redistributions of source code must retain the above copyright notice, this list of conditions and the following disclaimer. * Redistributions in binary form must reproduce the above copyright notice, this list of conditions and the following disclaimer in the documentation and/or other materials provided with the distribution. * Neither the name of Jaroslaw Kowalski nor the names of its contributors may be used to endorse or promote products derived from this software without specific prior written permission. THIS SOFTWARE IS PROVIDED BY THE COPYRIGHT HOLDERS AND CONTRIBUTORS "AS IS" AND ANY EXPRESS OR IMPLIED WARRANTIES, INCLUDING, BUT NOT LIMITED TO, THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE ARE DISCLAIMED. IN NO EVENT SHALL THE COPYRIGHT OWNER OR CONTRIBUTORS BE LIABLE FOR ANY DIRECT, INDIRECT, INCIDENTAL, SPECIAL, EXEMPLARY, OR CONSEQUENTIAL DAMAGES (INCLUDING, BUT NOT LIMITED TO, PROCUREMENT OF SUBSTITUTE GOODS OR SERVICES; LOSS OF USE, DATA, OR PROFITS; OR BUSINESS INTERRUPTION) HOWEVER CAUSED AND ON ANY THEORY OF LIABILITY, WHETHER IN CONTRACT, STRICT LIABILITY, OR TORT (INCLUDING NEGLIGENCE OR OTHERWISE) ARISING IN ANY WAY OUT OF THE USE OF THIS SOFTWARE, EVEN IF ADVISED OF THE POSSIBILITY OF SUCH DAMAGE.

NSubstitute

Copyright (c) 2009 Anthony Egerton (nsubstitute@delfish.com) and David Tchepek (dave@davesquared.net) All rights reserved. Redistribution and use in source and binary forms, with or without modification, are permitted provided that the following conditions are met: * Redistributions of source code must retain the above copyright notice, this list of conditions and the following disclaimer. * Redistributions in binary form must reproduce the above copyright notice, this list of conditions and the following disclaimer in the documentation and/or other materials provided with the distribution. * Neither the names of the copyright holders nor the names of its contributors may be used to endorse or promote products derived from this software without specific prior written permission. THIS SOFTWARE IS PROVIDED BY THE COPYRIGHT HOLDERS AND CONTRIBUTORS "AS IS" AND ANY EXPRESS OR IMPLIED WARRANTIES, INCLUDING, BUT NOT LIMITED TO, THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE ARE DISCLAIMED. IN NO EVENT SHALL THE COPYRIGHT OWNER OR CONTRIBUTORS BE LIABLE FOR ANY DIRECT, INDIRECT, INCIDENTAL, SPECIAL, EXEMPLARY, OR CONSEQUENTIAL DAMAGES (INCLUDING, BUT NOT LIMITED TO, PROCUREMENT OF SUBSTITUTE GOODS OR SERVICES; LOSS OF USE, DATA, OR PROFITS; OR BUSINESS INTERRUPTION) HOWEVER CAUSED AND ON ANY THEORY OF LIABILITY, WHETHER IN CONTRACT, STRICT LIABILITY, OR TORT (INCLUDING NEGLIGENCE OR OTHERWISE) ARISING IN ANY WAY OUT OF THE USE OF THIS SOFTWARE, EVEN IF ADVISED OF THE POSSIBILITY OF SUCH DAMAGE.

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"Contribution" shall mean any work of authorship, including the original version of the Work and any modifications or additions to that Work or Derivative Works thereof, that is intentionally submitted to Licensor for inclusion in the Work by the copyright owner or by an individual or Legal Entity authorized to submit on behalf of the copyright owner. For the purposes of this definition, "submitted" means any form of electronic, verbal, or written communication sent to the Licensor or its representatives, including but not limited to communication on electronic mailing lists, source code control systems, and issue tracking systems that are managed by, or on behalf of, the Licensor for the purpose of discussing and improving the Work, but excluding communication that is conspicuously marked or otherwise designated in writing by the copyright owner as "Not a Contribution."

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